

PACKAGING

Dental Dams:

Put the products into aluminium foil packaging and zip it closed, in order to protect rubber from worsening.

MAINTENANCE

- 1) Check a dirt, scar, bend and any other flaw before use.
- 2) Wash out extraneous matter such as blood, saliva and drug solution immediately after use. Do not use abrasives, scrubbers, metal wool and any metal tools for washing.
- 3) Remove remaining detergent and solution thoroughly under running water.
- 4) Completely dry out the instruments immediately after washing.
- 5) Replace the instruments in case there are signs of metal fatigue, worn out and other deteriorations.
- 6) " MB Rubber Dam Clamp set" can be steam-sterilized by storing in the accessory, "Sterilization Cassette S"

KSK Rubber Dam Punch
KSK Rubber Dam Clamp Forceps
Rubber Dam Template
Rubber Dam Frame

Adult
Pedo

Dental Dam

Medium
Thin
Non-Latex
Non-Latex Powder-Free

Rubber Dam Clamp Board with Clamps

No.8 Normal
No.8 Matt
No.8 Black
No.12 Normal
No.12 Matt
No.12 Black
No.18 Normal
No.18 Matt
No.18 Black
I-Series Normal
I-Series Matt
I-Series Black
S-Series Normal
S-Series Matt
S-Series Black

Rubber Dam Clamp Board with Deciduous Clamps

Normal, Matt, Black

MB Rubber Dam Clamp set

Rubber Dam Technic Set

Normal, Matt, Black

Rubber Dam Starter Kit

Rubber Dam Clamps

Molar: 15, 18, 24, 25, 26, 28, 30, 31, 42, 43, 51,
138, 139, 200, 201, 202, 203, 204, 205, 3, W3, 4, 5,
56, 56A, 7, 7A, 7B, 8, 8A, W8A, 10, 11, 12A, 13A, 14, 14A, 14AN,
MB1, MB2, MB4
Bicuspid: 22, 27, 29, 206, 207, 208, 209, 1, 1A, 2, 2A, W2A, 0,
00, MB5
Labial: 210, 211, 212, 212SA, 6, 9, 44, 90N
Deciduous: P-1, P-2, 54, 55, 64, 65, 74, 75, 84, 85, MB3
Tiger: 1T, 1AT, 2T, 2AT, 14T, 56T, 9T
Others: G-1, G-2, 214, U67

Rubber Dam Clamp Boards (without clamps)

No.8, No.12, No.18, I-Series, S-Series, Deciduous

Sterilization Cassette S

REF 1021011

REF 1031040

REF 08190020

REF 1022010

REF 1022020

REF 1023020

REF 1023030

REF 1023040

REF 1023050

REF 1040000

REF 1040080

REF 1040050

REF 1040010

REF 1040090

REF 1040060

REF 1040020

REF 1040100

REF 1040070

REF 1043001

REF 1043005

REF 1043003

REF 1043002

REF 1043006

REF 1043004



KSK

RUBBER DAM INSTRUMENTS

Instructions for Use

WARNING NOTICES

Clamps:

Don't broaden width of Clamps over +1.5mm.

Dental Dams:

Available only for single use. Do not reuse.



Dental Dam Medium and Dental Dam Thin:

The products are made of natural rubber. Natural rubber rarely causes allergic symptoms such as itching, redness, urticaria, swelling, fever, dyspnoea, asthma-like symptoms, blood pressure lowering, shock. In case of such symptoms, stop using it immediately and consult a doctor.



INDICATIONS

For dental rubber dam isolation.

NOTES

No special notices.

DIRECTIONS FOR USE

Must autoclave "Clamps", "KSK Rubber Dam Punch", "KSK Rubber Dam Clamp Forceps" and "Rubber Dam Frame" before use.

- 1) Confirm the suitability of rubber dam clamps while being careful not to injure the gums.
- 2) Mark a hole position on the rubber dam using rubber dam template.
- 3) Select a hole on the turret of rubber dam punch depending on the size of a diseased tooth.
- 4) Punch the rubber dam with the rubber dam punch.
- 5) Pass the rubber dam clamp through the dam's

opening and place it parallel to imaginary dental arch on the rubber dam.

- 6) Hold the rubber dam clamp with a rubber dam forceps folding the rubber dam.
 - 7) Place the rubber dam clamp onto a diseased tooth.
 - 8) Slide the rubber dam below the rubber dam clamp and stick it firmly to approximal surfaces of the teeth using dental floss.
 - 9) Position the rubber dam frame while be careful not to prevent patients breathing.
- ※ When using rubber dam for a long time, you can use mouse prop in combination. [To reduce the burden on the temporomandibular joint]

DISINFECTION

KSK Rubber Dam Punch, KSK Rubber Dam Clamp Forceps, Rubber Dam Frame, Rubber Dam Clamps and Sterilization Cassette S:

Use disinfecting solution excepting chlorine, iodine-based disinfect solution, sodium hypochlorite, benzalkonium chloride, povidone iodine, formalin, phenol, chlorhexidine gluconate, strong acidic water and household cleaners.

Rubber Dam Template:

Use disinfectant alcohol.

STERILIZATION

KSK Rubber Dam Punch, KSK Rubber Dam Clamp Forceps, Rubber Dam Frame, Rubber Dam Clamps, Boards and Sterilization Cassette S:

Steam sterilization at 132°C for 5 minutes or 121°C for 15 minutes according to ISO17665-2.

Dental Dams and Rubber Dam Template:

Not autoclavable.

STORAGE

Do not store together with rusted products and chemical medicine.

Rev.5-5.2019



Manufacturer:



DENTECH Corporation
53-5, Shimizu-cho, Itabashi-ku
Tokyo, 174-0053, Japan
TEL +81-3-3962-5621
FAX +81-3-3962-5624



www.dentech.co.jp

EC REP

MDSS GmbH

Schiffgraben 41
30175 Hannover, Germany