



CATALOGUE

English

2021



History of DENTECH

In 1907, Dentech Corporation, a pioneering manufacturer in Dental Instruments, was founded in Yanaka, Tokyo by Kenjiro Kimura, and it was originally named as Kimura Kou-sakusho.

We manufactured the first Japan made dental burs.

After Second World War

In 1944, the first factory was destroyed due to Second World War. In 1947, after the war, we transferred our factory to Itabashi in Tokyo and reconstructed it. However, in 1954, the strongest ever typhoon unfortunately struck around the area and the factory collapsed again. In 1955, we rebuilt a new factory at the place nearby the destroyed factory to continue providing our products for the customers.

First Advancement Overseas

After Second World War, a lot of dentists came to Japan from the U.S. accompanying the occupation army and needed trustworthy dental instruments with high quality. Thus we were able to develop our products through the cooperation of them. In 1971, we launched to export our products overseas and our first overseas customer was in the United States.

Renamed as DENTECH

Kimura Kosakusho was renamed as Dentech Corporation in 1974. Since 1976 we have manufactured drills for implantology as the first implant drill manufacturer in Japan. Nowadays most Japanese implant makers adopt our drills.

The Current DENTECH

Today Dentech is well known as one of the companies which have the longest over 100 - year history in dental industry. KSK rubber dam instruments, implantation drills and burs as well as many unique products with Japan-made high quality are well supported by dentists and professionals all over the world.

Joint Performance with Japanese CRAFTSMANSHIP and Latest Machine Technologies

We possess CNC lathe, CNC Grinding machine, Heat treating machine, 3D CAD/CAM systems, and other various latest machines and equipments at our own factory in Japan. At the same time we are proud of the greatest technique by our craftsmen whose skill is unequaled with any others. We have an advantage in manufacturing extremely delicate dental/medical instruments using these two wheels which are identified by the machine automation and craftsmen's technique.

We have been developing, designing and manufacturing any kinds of dental and surgical instruments needed in clinical site around the world with outstanding quality for over a century.

It is a great honor for us that our products are highly respected all over the world. Our products would surely satisfy you.

Dentech has been assessed and certified as meeting the requirements of ISO 13485.



KSK Rubberdam system's products has verified for the European CE mark medical device regulation.



CONTENTS

ENDODONTICS P.6

KSK Rubber Dam Clamps	P.6	Peeso Reamers	P.23
Dental Dams	P.17	RTP Reamers	P.23
Rubber Dam Accessories	P.18	Finger Ruler	P.24
Rubber Dam Set	P.21	X-Ray Mesh Gauges	P.24
M Peeso Reamers	P.23	Resin Post Methacrylics	P.24
		Retro Filling Retractor	P.25

PROSTHETICS P.26

Matrix Bands	P.26	Tooth Separators	P.27
Matrix Retainer	P.26	Articulating Paper Holders	P.28
Tooth Contact Gauges	P.27	New Fine Lock Trays	P.29
New Inlay & Crown Setters	P.27		

IMPLANTOLOGY P.30

Twist Drills Black	P.30	Drill Extenders	P.36
Step Drills For Guide	P.31	Drill Extension SHORT	P.36
Twist Drills For Guide	P.31	Para Drill Aid II	P.37
Twist Drills	P.32	Technika Bone Mill Set	P.37
Trephine Burs	P.33	Technika Repair Tap	P.38
Bone Trephines	P.33	Depth Gauges	P.38
Circuler Knives	P.33	3.75 Special Drills	P.38
Surgical Burs	P.34	Gingiva Gauge	P.39
Technika TK Drill System	P.35	Sterilization Box	P.39
		Bur Stands	P.39

LABORATORY P.40

Vulcanite Burs	P.40	Air Jet Diamonds	P.40
----------------	------	------------------	------

MISCELLANEOUS P.41

Lip & Cheek Retractor	P.41	Parallel Reflector	P.43
Dry Field Mouth Props	P.41	pinlight2	P.44
Lip Expanders	P.41	Excavators	P.45
Bite Expander	P.42	Wax Spatulas 7A	P.45
Big Oral Mirrors Real-Fine	P.42	Evans	P.46
Cotton Pliers	P.43	Hollenback #3	P.46
Tongue Guards & Saliva Ejector	P.43	Woodson #1	P.46

Autoclavable

			
Up to 121°C for 15 min	Up to 132°C for 5 min	Non - autoclavable	Made-to-order items

DENTECH Corporation (copyright, all rights reserved).
 The Catalogue is intended to the practicing dentists, students of the medical universities and colleges, dental trade dealers.
 Design of the Catalogue, together with all the images, texts, graphic elements, text markers and identifiers are intellectual property of our company.
 For partial or full copying or loaning of any of the above mentioned elements for commercial purposes, you are obliged to ask for it and receive written permission.

Catalogue is for information only. The company does not carry any liability for possible miss prints, text mistakes or difference in products' colour or form of the items pictured on the images. The company leaves its' right for changes without prior consent.

ENDODONTICS


PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

KSK Rubber Dam Clamps

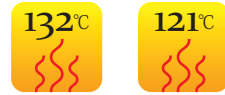

Features

- KSK Rubber Dam Clamps are made of specially selected and tempered stainless steel.
- Ideal spring action and high resistance against the corrosion.
- Various forms are available, so you can choose clamps in accordance with the teeth.
- 4000 times open-close metal fatigue test goes for every production lot to assure long-term spring action.
- For stable forms, our craftsmen adjust bending every KSK clamps one by one manually.
- All beaks are sharpened for best fitting on the teeth.
- "MATT" and "BLACK" can avoid eye fatigue of dentists from reflecting operation light.

Finishing



GLOSSY
Glossy finishing.




MATT
Sand blasted non glossy finishing.




BLACK
Not painted, Oxide layer finishing which is safe for human body

Winged or Wingless

WINGED

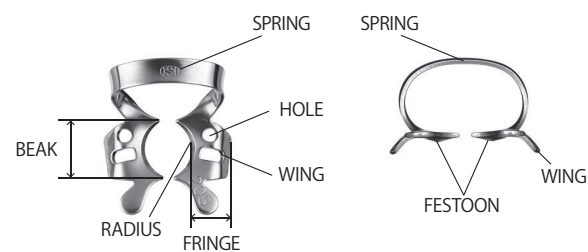

WINGLESS

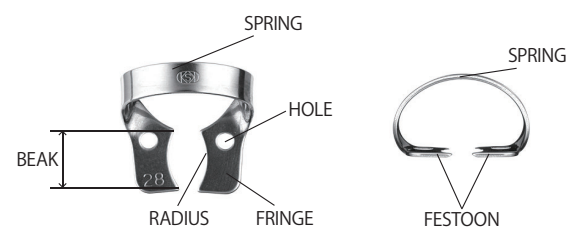

[Important Safety Instructions]

It is likely to transform and to break when force is excessively added.
Please do not expand rubber dam clamps more than the necessity.

KSK Rubber Dam Clamps

Molar Clamps

Product Code

- Glossy
- Matt
- Black

3

- 01012003
- 01014003
- 01019003

For small lower molars with flat beaks.

W3

- 01012903
- 01014903
- 01019903

For small lower molars with flat beaks. Similar to 3 but wingless.

4

- 01012004
- 01014004
- 01019004

For upper molars with festooned beaks.

5

- 01012005
- 01014005
- 01019005

For upper molars with festooned beaks.

7

- 01012007
- 01014007
- 01019007

For big lower molars with flat beaks.

7A

- 01012907
- 01014907
- 01019907

For big lower molars with flat beaks.

8

- 01012008
- 01014008
- 01019008

For big upper molars with festooned beaks. Same size as 7.

8A

- 01012908
- 01014908
- 01019908

For upper molars with deeply festooned beaks which can grab subgingival part.


Molar Clamps

Product Code

- Glossy
- Matt
- Black

W8A


- 01012998
- 01014998
- 01019998

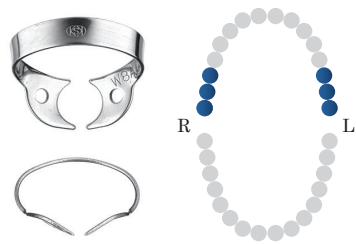

Similar to 8A but wingless.

10 *Discontinued


- 01012010
- 01014010
- 01019010

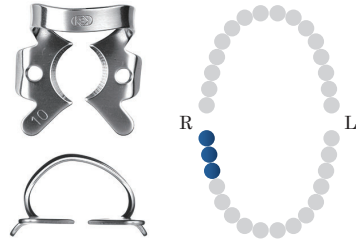

<STOCK ONLY>
recommended similar type #203

11 *Discontinued


- 01012011
- 01014011
- 01019011

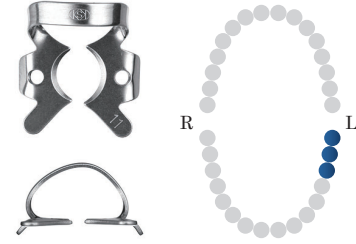

<STOCK ONLY>
recommended similar type #204

12A


- 01012912
- 01014912
- 01019912

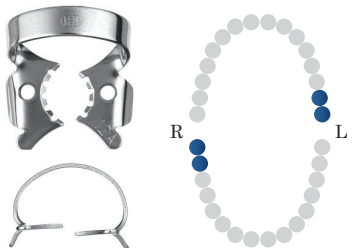

Especially for the third upper left and lower right molars with flat serrated beaks, paired with 13A.

13A


- 01012913
- 01014913
- 01019913

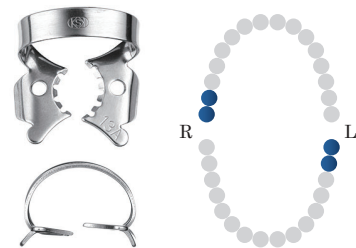

Especially for the third lower left and upper right molars with flat serrated beaks, paired with 12A.

14


- 01012014
- 01014014
- 01019014

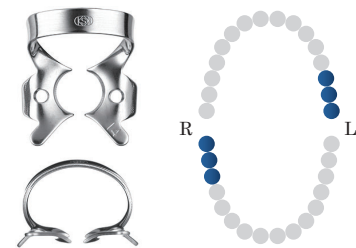

For partially erupted molars with deeply festooned beaks.

14A

- 01012914
- 01014914
- 01019914

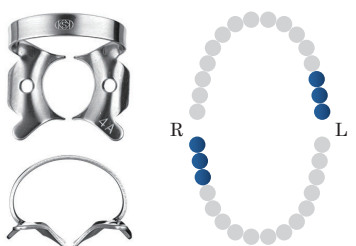

For partially erupted molars with deeply festooned beaks, bigger than 14.

15


- 01012015
- 01014015
- 01019015



For big lower molars with flat and longer beaks, a strong spring.

18

- 01012018
- 01014018
- 01019018

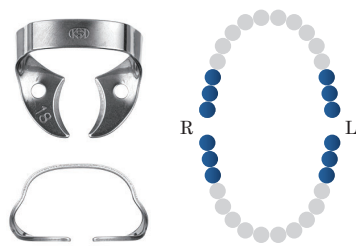

For upper & lower molars with flat beak.

Molar Clamps

Product Code

- Glossy
- Matt
- Black

24

- 01012024
- 01014024
- 01019024



For upper left and lower right buccal molars' subgingival cavities. The buccal side has wider fringe slightly festooned beaks. Paired with 25.

25

- 01012025
- 01014025
- 01019025



For lower left and upper right buccal molars' subgingival cavities. The buccal side has wider fringe slightly festooned beaks. Paired with 24.

26

- 01012026
- 01014026
- 01019026



For upper molars in case of exposure of the tooth neck with slightly festooned beaks. Available for the second bicuspids.

28

- 01012028
- 01014028
- 01019028



For upper and lower molars with flat beaks, wide view.

30


- 01012030
- 01014030
- 01019030



For lower left molar subgingival with slightly festooned beaks. The projection on the buccal beak slips into the furcation. Paired with 31.

31


- 01012031
- 01014031
- 01019031

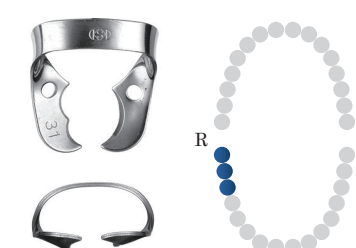

For lower right molar subgingival with slightly festooned beaks. The projection on the buccal beak slips into the furcation. Paired with 30.

42


- 01012042
- 01014042
- 01019042

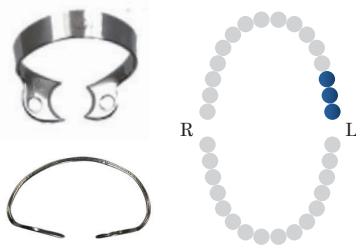

For upper left molars with flat beaks, paired with 43.

43

- 01012043
- 01014043
- 01019043

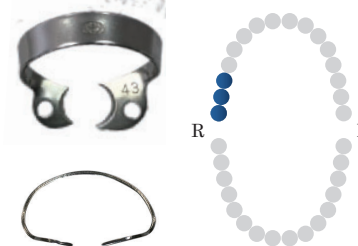

For upper right molars with flat beaks, paired with 42.

51

- 01012051
- 01014051
- 01019051




For buccolabial molar furcations with slightly festooned beaks. Enables to press down the gingiva.

Molar Clamps

Product Code
■ Glossy
■ Matt
■ Black

56


01012056
 01014056
 01019056

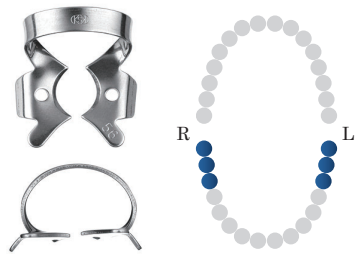

For lower molars with with festooned beaks.

56A


01012956
 01014956
 01019956

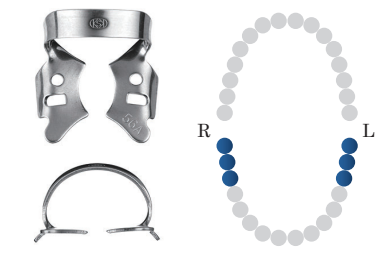

For big lower molars with very slightly festooned beaks. One size larger than 56.

U67

01012067
 01014067
 01019067

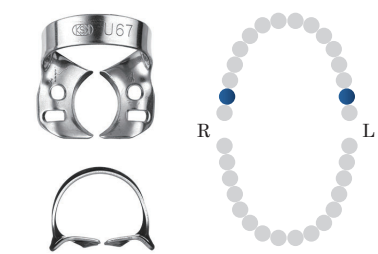


Best for upper second molars with festooned beaks, compact size.

Molar Clamps

Product Code
■ Glossy
■ Matt
■ Black

205


01012205
 01014205
 01019205

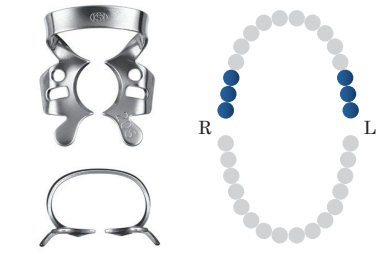

For upper molars with very slightly festooned beaks.

G-1


50580002
 50550010
 50550030

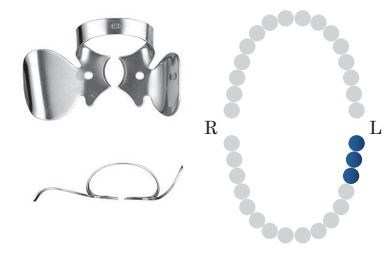

For lower left molars with cheek and tongue pressure shoving wings. No need to wear dental dam but gauze or cotton rolls. Paired with G-2.

G-2


50580003
 50550011
 50550031

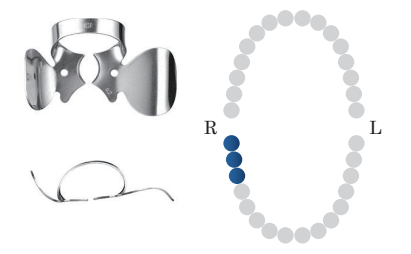

For lower right molars with cheek and tongue pressure shoving wings. No need to wear dental dam but gauze or cotton rolls. Paired with G-1.

138


01012138
 01014138
 01019138

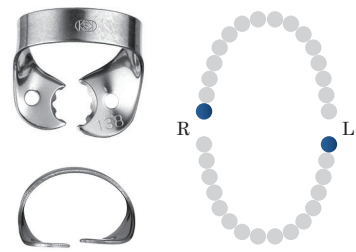

For upper right and lower left third molars with serrated offset beaks, paired with 139.

139


01012139
 01014139
 01019139

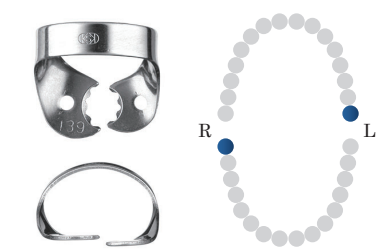

For upper left and lower right third molars with serrated offset beaks, paired with 138.

201

01012201
 01014201
 01019201

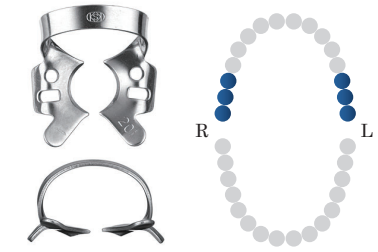

For upper molars with slightly festooned beaks.


KSK Rubber Dam Clamps

Bicuspid Clamps

Product Code
■ Glossy
■ Matt
■ Black

0


01012701
 01014701
 01019701

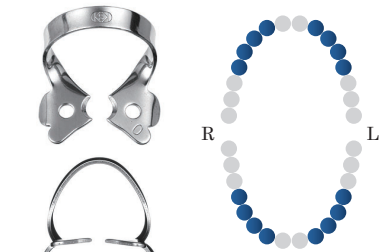

For narrow bicuspid cervical zone with flat beaks. Available for deciduous anteriors as well.

00


01012702
 01014702
 01019702

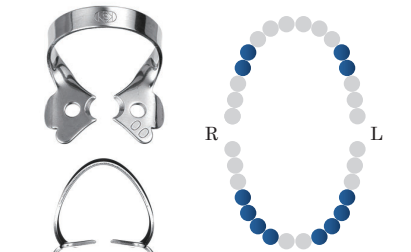

For small bicuspid with flat beaks. Available for deciduous anteriors as well.

202


01012202
 01014202
 01019202

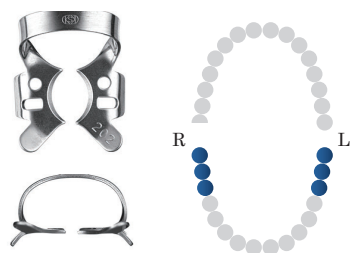

For big lower molars with flat beaks. A little bigger than 200.

203


01012203
 01014203
 01019203

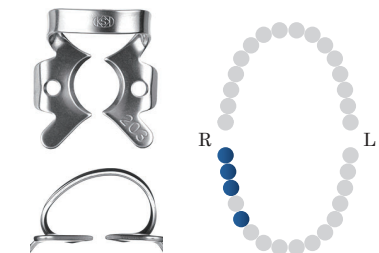

For lower right molars and the first bicuspid with flat beaks, Paired with 204.

204


01012204
 01014204
 01019204

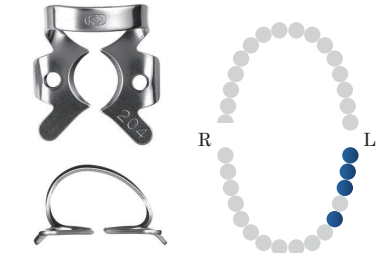

For lower left molars and the first bicuspid with flat beaks, Paired with 203.

1


01012001
 01014001
 01019001

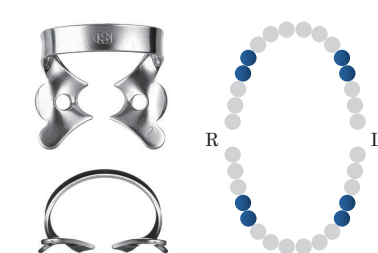

For bicuspid root with festooned beaks.

1A

01012901
 01014901
 01019901

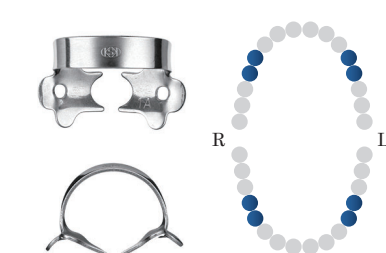

For cervical cavities of bicuspid with festooned beaks.

2

01012002
 01014002
 01019002



For lower bicuspid with flat beaks.

ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

Bicuspid Clamps

Product Code
■ Glossy
■ Matt
■ Black



For bigger upper and lower bicuspid with flat beaks. A little bigger than 2.



Similar to 2A but wingless.



For upper and lower bicuspid with flat beaks, similar to 207 but wingless.




For upper and lower bicuspid with festooned beaks, similar to 206 but wingless.




For upper and lower bicuspid with flat beaks, for wider view.




For upper and lower bicuspid with festooned beaks. Similar to 27 but winged.

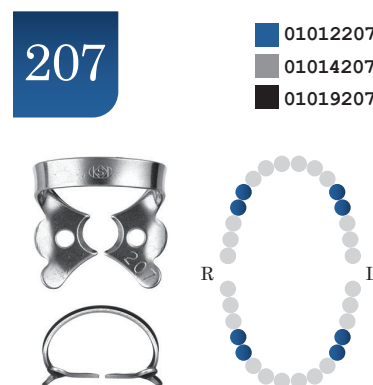

For upper and lower bicuspids with flat beaks. Similar to 22 but winged.

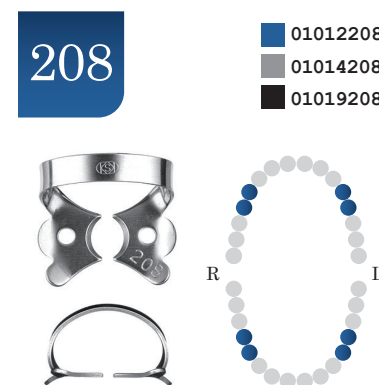



For big bicuspid. Similar to 207 but the fringe is little bigger.

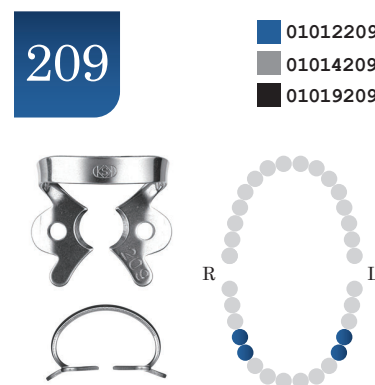


For lower bicuspid with flat beaks. Similar to 208 but a little smaller and stronger spring.

Labial Clamps

Product Code
■ Glossy
■ Matt
■ Black

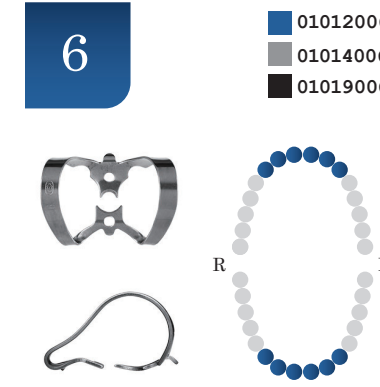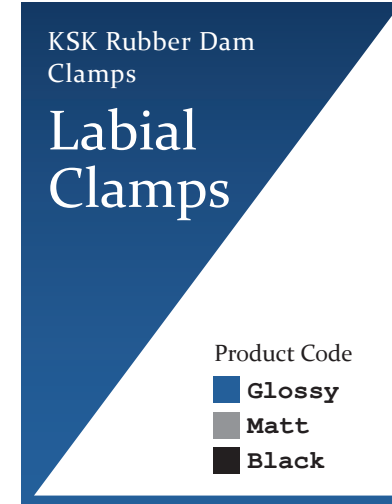


For anteriors with slightly offset beaks.

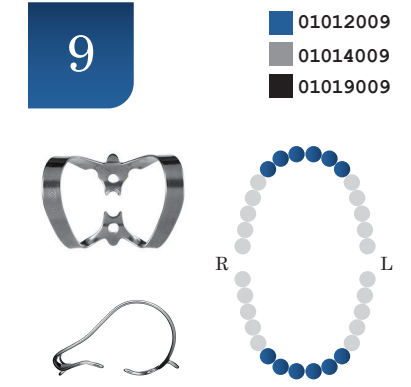


For labial cavities on anteriors with slightly offset beaks.



For anteriors cuspid and cavities with flat beaks.

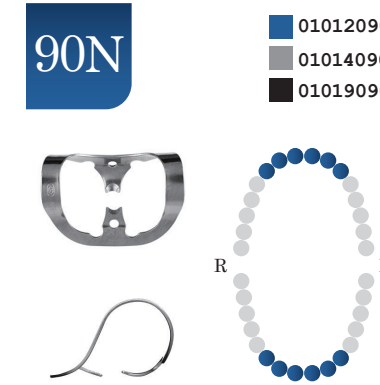

Offset beaks and stronger spring.

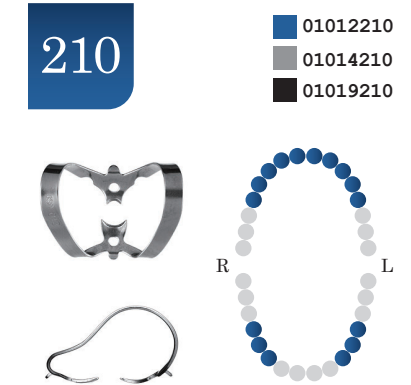

For anteriors labial and canines and cavities on bicuspid labial.



For gum compression on anteriors with offset beaks.



Similar to 212 but a little stronger pressure to gingiva.


Deciduous Clamps

Product Code
■ Glossy
■ Matt
■ Black

KSK Rubber Dam Clamps
Deciduous Clamps


Product Code
■ Glossy
■ Matt
■ Black

54
■ 01013001
■ 01015001
■ 01020001

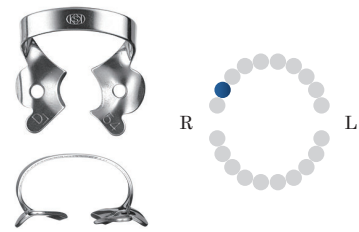

For upper right the first deciduous molars. (B/54)

55
■ 01013005
■ 01015002
■ 01020002

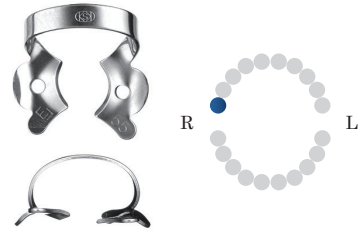


For upper right the second deciduous molars. (A/55)

Deciduous Clamps


Product Code
■ Glossy
■ Matt
■ Black

P-1
■ 50580000
■ 50550000
■ 50550020

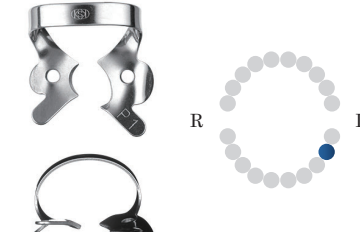


For lower left the first deciduous molars. (L/74)
 Paired with P-2.

P-2
■ 50580001
■ 50550001
■ 50550021

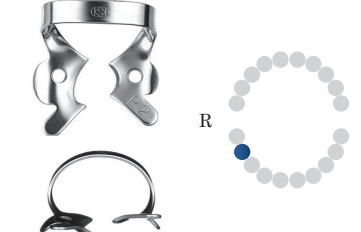


For lower right the first deciduous molars. (S/84)
 Paired with P-1.

64
■ 01013002
■ 01015003
■ 01020003

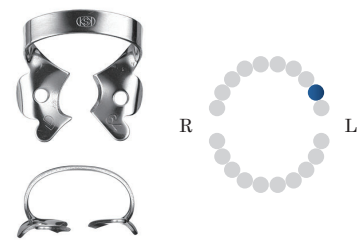


For upper left the first deciduous molars. (I/64)

65
■ 01013006
■ 01015004
■ 01020004

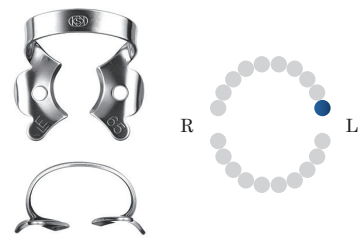


For upper left the second deciduous molars. (J/65)

74
■ 01013003
■ 01015005
■ 01020005

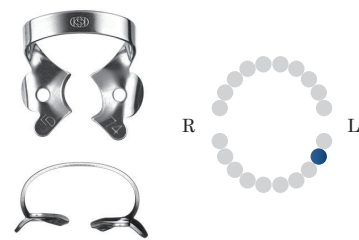

For lower left the first deciduous molars. (L/74)

75
■ 01013007
■ 01015006
■ 01020006

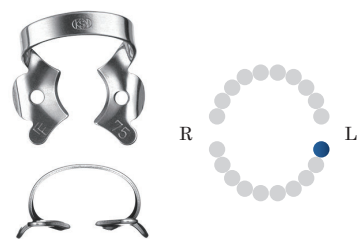


For lower left the second deciduous molars. (K/75)

84
■ 01013004
■ 01015007
■ 01020007



For lower right the first deciduous molars. (S/84)

85
■ 01013008
■ 01015008
■ 01020008

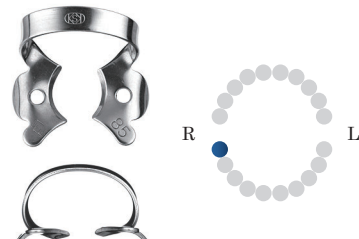

For lower right the second deciduous molars. (T/85)

KSK Rubber Dam Clamps

Tiger Clamps

Product Code
■ **Glossy**
■ **Matt**
■ **Black**

1T

■ 01012501
■ 01014501
■ 01019501

For bicuspid root with festooned serrated beaks.

1AT

■ 01012502
■ 01014502
■ 01019502

For bicuspids, especially for subgingival cavities with deeply festooned serrated beaks.

2T

■ 01012503
■ 01014503
■ 01019503

For lower bicuspids with flat serrated beaks.

2AT

■ 01012504
■ 01014504
■ 01019504

For upper and lower bigger bicuspids with flat serrated beaks. A little bigger than 2T

9T

■ 01012505
■ 01014505
■ 01019505

Used for labial cavities on anteriors with offset serrated beaks.

14T

■ 01012506
■ 01014506
■ 01019506

For partially erupted molars with serrated deeply festooned beaks.

56T

■ 01012507
■ 01014507
■ 01019507

For lower molars with festooned beaks.

Dental Dams


Dental Dams (Pure natural rubber)

- Features**
- KSK dental dams have a good resistance and elastic property.
 - Packed in airtight aluminium bags to keep the rubber sheet fresh.
 - "Thin" is made of color free material then capable for monitoring the condition of an patient during isolation treatment.

[Specification]
 Packaging : 36 sheets / pack Size : 6" x 6" (152mm x 152mm)
 Thickness : 0.20mm(Medium), 0.18mm(Thin) Color : Green(Medium), Ivory(Thin)
 Scent : Non

Product Code 01023020
 Product **KSK Dental Dam Medium (cut)6"x6"**

Product Code 01023030
 Product **KSK Dental Dam Thin (cut)6"x6"**

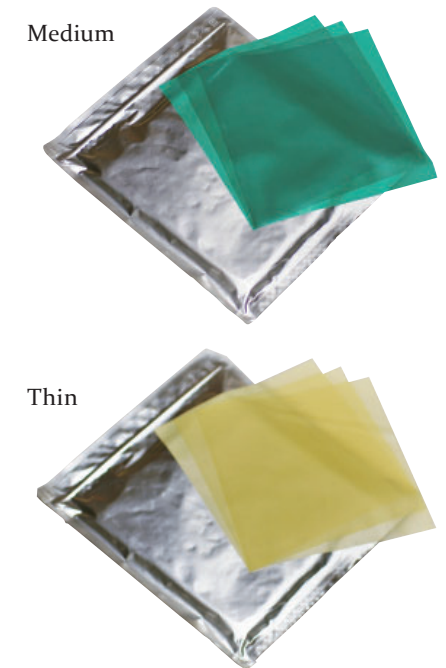

Dental Dam Non-Latex

- Features**
- Used when there is a concern regarding an individual's possible allergic reaction to the proteins present in natural rubber.
 - Packed in airtight aluminium bags to keep the rubber sheet fresh.

[Specification]
 Packaging : 20 sheets / pack Size : 6" x 6" (152mm x 152mm)
 Thickness : 0.20mm Color : Blue Scent : Non

Product Code 01023040
 Product **KSK Dental Dam (cut)6"x6" Non-Latex**



Dental Dam Non-Latex Powder-Free

- Features**
- Powder-free minimizes the risk of illness or injury caused by allergy for all dental health care personnel and patients.
 - Packed in airtight aluminium bags to keep the rubber sheet fresh.

[Specification]
 Packaging : 15 sheets / pack Size : 6" x 6" (152mm x 152mm)
 Thickness : 0.25mm Color : Purple Scent : Mint

Product Code 01023050
 Product **KSK Dental Dam (cut)6"x6" Non-Latex Powder-Free**



Rubber Dam Accessories

KSK Rubber Dam Punch

[Specification]
 Material : Stainless steel
 Size : 153 x 70mm
 Type : Ainsworth type.
 Hole diameter : 1.4mm , 1.6mm , 1.8mm , 2.1mm , 2.3mm

Product Code 01021011
 Product KSK Rubber Dam Punch



KSK Rubber Dam Clamp Forceps

Features

Original design of DENTECH Rubber Dam Clamps are automatically off from the forceps at the place of operator fits them on tooth.

[Specification]
 Material : Stainless steel
 Size : 165 x 70mm


Product Code 01031040
 Product KSK Rubber Dam Clamp Forceps



Rubber Dam Frames Young Type

[Specification]
 Material : Stainless steel
 Adult : 108(w) x 95mm(h)
 Pedo : 77(w) x 73mm(h)

Product Code 01022010
 Product Rubber Dam Frame, Adult
 Product Code 01022020
 Product Rubber Dam Frame, Pedo

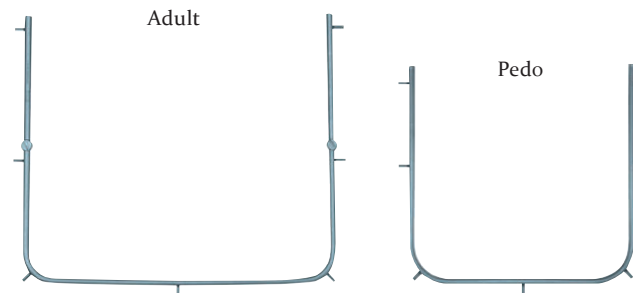


Rubber Dam Template

Features

For marking punch holes.

[Specification]
 Material : Plastic
 Size : 150 x 150mm

Product Code 08190020
 Product Rubber Dam Template

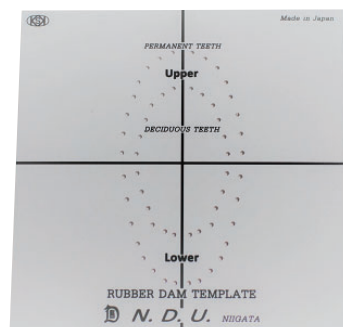

Clamp set

Clamp set No.8

[Specification]
 Material : Stainless steel Size : 90 x 90mm
 With 8 clamps : #9 x 2, 2A, W2A, 5, 56, 203 & 204 or Clamp Organizer (Board only)

Product Code 01041000 Product Clamp Organizer
 Product Code 01040000 Product Glossy
 Product Code 01040080 Product Matt
 Product Code 01040050 Product Black



Clamp set No.12

[Specification]
 Material : Stainless steel Size : 150 x 90mm
 With 12 clamps : #210, 9, 0, 2, 2A, 206, 207, 5, 56, 7, 201 & 202 or Clamp Organizer (Board only)


Product Code 01041010 Product Clamp Organizer
 Product Code 01040010 Product Glossy
 Product Code 01040090 Product Matt
 Product Code 01040060 Product Black



Clamp set No.18

[Specification]
 Material : Stainless steel Size : 160 x 120mm
 With 18 clamps : #210, 9, P-1, P-2, 22, 27, 29, 208, 209, 18, 138, 139, 7, 201, 202, 203, 204 & 205 or Clamp Organizer (Board only)

Product Code 01041020 Product Clamp Organizer
 Product Code 01040020 Product Glossy
 Product Code 01040100 Product Matt
 Product Code 01040070 Product Black

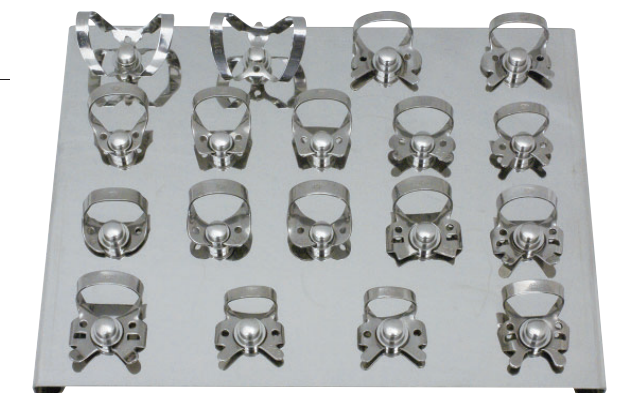



Deciduous Clamp set

[Specification]

Material : Stainless steel Size : 90 x 90mm
Complete kit of 8 patterns with special Clamp board.
#55-upper right E #54-upper right D #64-upper left D #65-upper left E
#85-lower right E #84-lower right D #74-lower left D #75-lower left E

Product Code	01041061	Product	Clamp Organizer
Product Code	01041060	Product	Glossy
Product Code	01041070	Product	Matt
Product Code	01041080	Product	Black



Clamp set with cassette

[Specification]

Material : Stainless steel Size : 94 x 94 x 34 mm
Clamp set No.8 or Deciduous Clamp set with sterilization cassette

Product Code	01040001	Product	No.8, Glossy
Product Code	01040081	Product	No.8, Matt
Product Code	01040051	Product	No.8, Black
Product Code	01041062	Product	Deciduous, Glossy
Product Code	01041071	Product	Deciduous, Matt
Product Code	01041081	Product	Deciduous, Black



Rubber Dam Set




Rubber Dam Technic set

[Specification]

Contents : ①Clamp set No.12
②Rubber Dam Frame Large(Adult)
③Rubber Dam Punch(Ainsworth)
④Rubber Dam Clamp Forceps
Clamps : #210, #9, #0, #2, #2A, #206, #207, #5, #56, #7, #201, #202

Product Code	01030100	Product	Glossy
Product Code	01030120	Product	Matt
Product Code	01030110	Product	Black

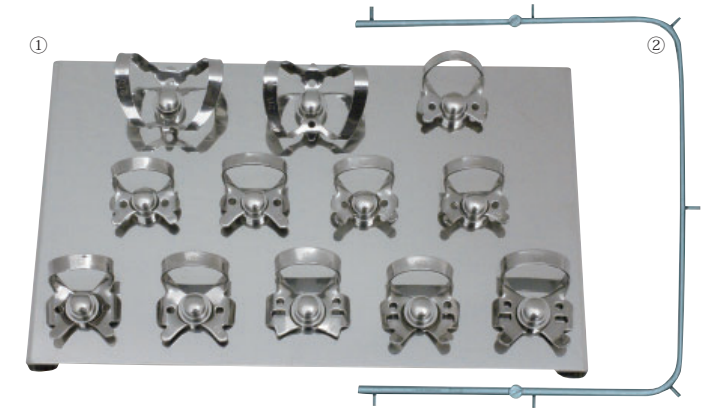


Rubber Dam Complete set

[Specification]

Material : Stainless steel Size : 274 x 184 x 34 mm
Rubber Dam Technic set with sterilization cassette

Product Code	01030101	Product	Glossy
Product Code	01030121	Product	Matt
Product Code	01030111	Product	Black

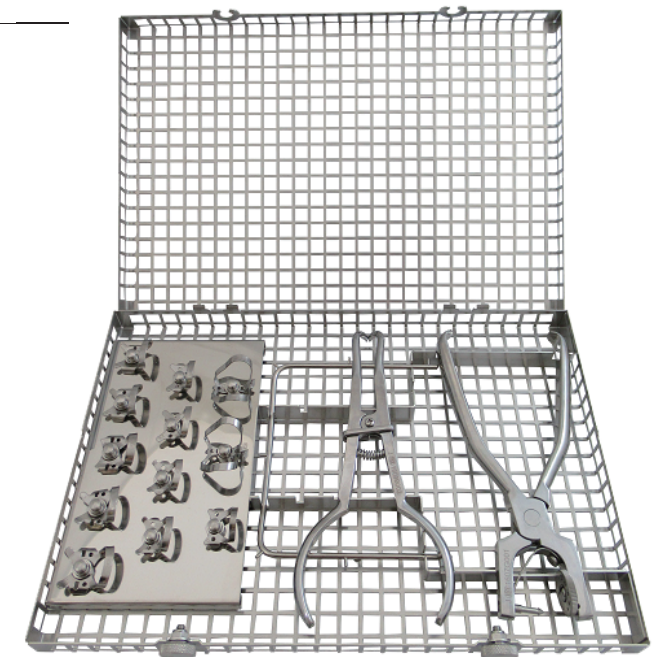





Rubber Dam Starter Kit

[Specification]


Basic assortment for rubberdam technique

- Contents :
- ① Clamp set No.12,
 - ② Rubber Dam Frame(Adult)
 - ③ KSK Rubber Dam Punch(Ainsworth)
 - ④ KSK Rubber Dam Clamp Forceps
 - ⑤ KSK Dental Dam Non-Latex Blue(20 sheets)
 - ⑥ Rubber Dam Template

Product Code	01030100	Product	Glossy
Product Code	01030120	Product	Matt
Product Code	01030110	Product	Black

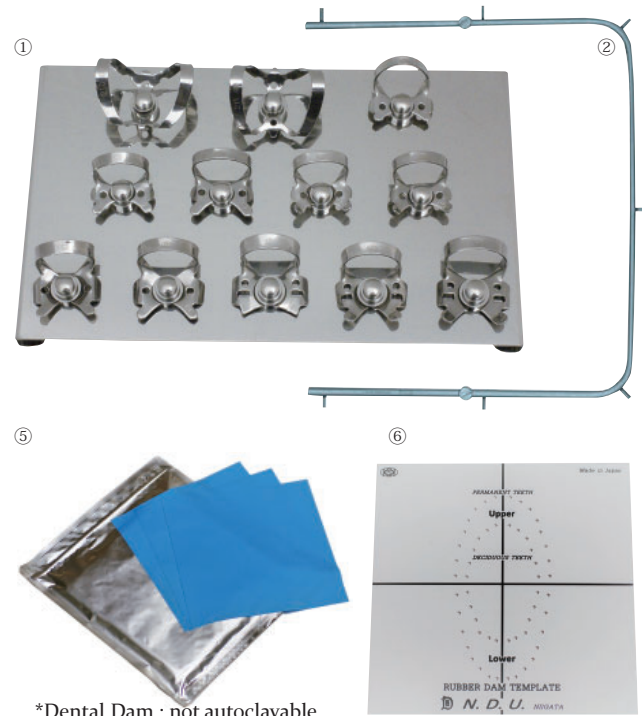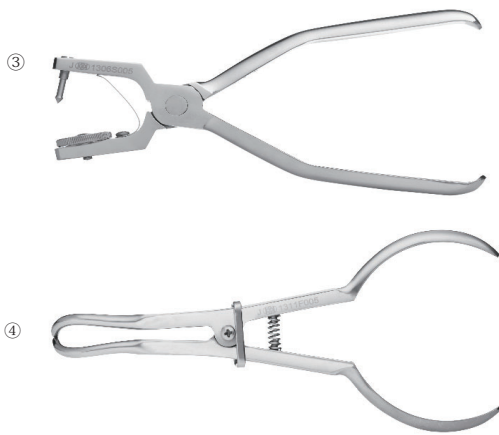

*Dental Dam : not autoclavable

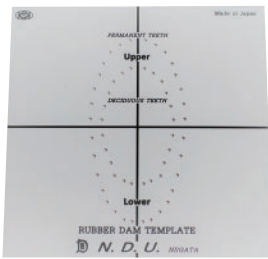

M Peeso Reamers


[Specification]

Material : Stainless steel

Sharp shaped top

Pilot hole drills for root canal treatment.

Product Code	02312200	Product	M Peeso Reamer CA #000 (set of 4 pcs)
Product Code	02312201	Product	M Peeso Reamer CA #00 (set of 4 pcs)
Product Code	02312202	Product	M Peeso Reamer CA #0 (set of 4 pcs)
Product Code	02312203	Product	M Peeso Reamer CA #1 (set of 4 pcs)
Product Code	02312204	Product	M Peeso Reamer CA #2 (set of 4 pcs)
Product Code	02312205	Product	M Peeso Reamer CA #3 (set of 4 pcs)

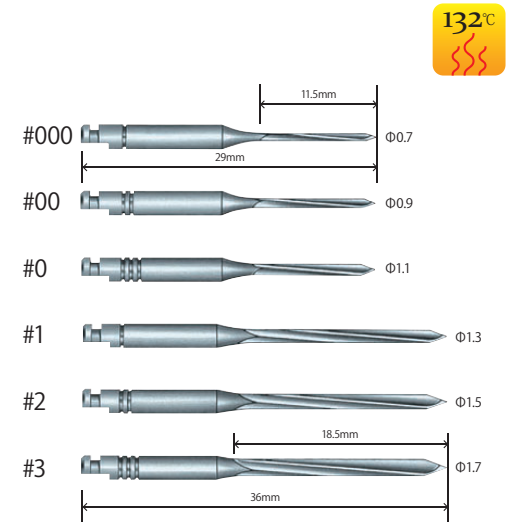

Peeso Reamers


[Specification]

Material : Stainless steel

Round shaped top

Pilot hole drills for root canal treatment.

Product Code	02312000	Product	Peeso Reamer CA #000 (set of 4 pcs)
Product Code	02312001	Product	Peeso Reamer CA #00 (set of 4 pcs)
Product Code	02312002	Product	Peeso Reamer CA #0 (set of 4 pcs)
Product Code	02312003	Product	Peeso Reamer CA #1 (set of 4 pcs)
Product Code	02312004	Product	Peeso Reamer CA #2 (set of 4 pcs)
Product Code	02312005	Product	Peeso Reamer CA #3 (set of 4 pcs)

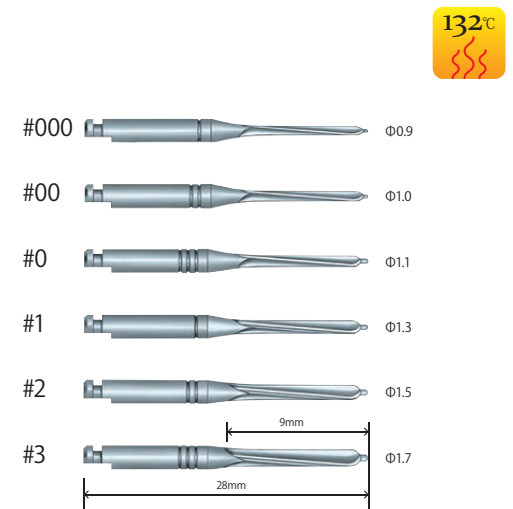

RTP Reamers


[Specification]

Material : Stainless steel

Round shaped

Final drills for root canal treatment.

Product Code	02312100	Product	RTP Reamer #1 (set of 4 pcs)
Product Code	02312101	Product	RTP Reamer #2 (set of 4 pcs)
Product Code	02312102	Product	RTP Reamer #3 (set of 4 pcs)

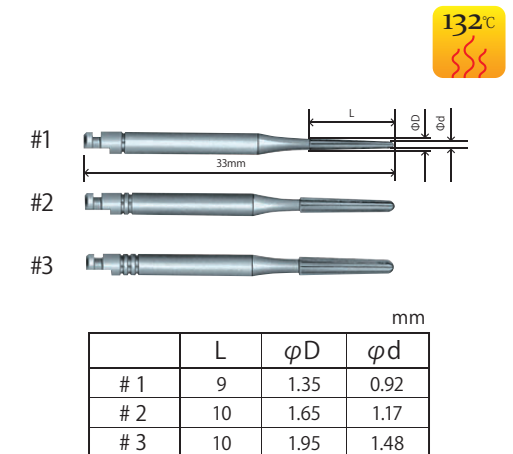

	L	φD	φd
# 1	9	1.35	0.92
# 2	10	1.65	1.17
# 3	10	1.95	1.48



Finger Ruler

Description


Can be attached on finger by free size. The adjustable stopper keeps its position by spring.

[Specification]

Material : Stainless steel Measurement length : 32mm

Product Code 03050000

Product **Finger Ruler**

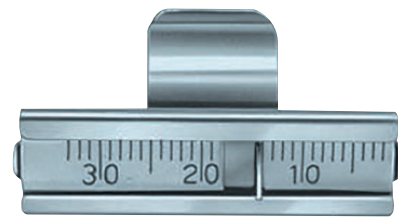

X-Ray Mesh Gauge

Description

Suitable for root canal treatment. Accurate examination before placing an implant. checking of calculus quantity and its area.

[Specification]

Cover Material : Polyester Mesh Line Material : Stainless steel

•Dental Size

Size : 40 x 30mm Fine Grid : pitch 1mm Wide Grid : pitch 5mm

•Panorama Size



Size : 300 x 150mm Fine Grid : pitch 2mm Wide Grid : pitch 10mm

Product Code 07050010

Product **X-Ray Mesh Gauge, Dental**

Product Code 07050011

Product **X-Ray Mesh Gauge, Panorama**

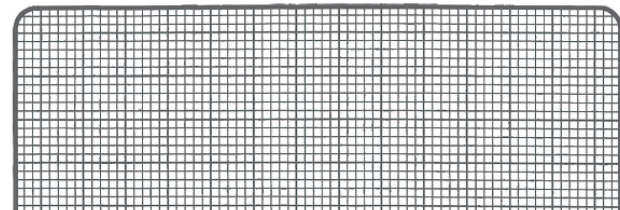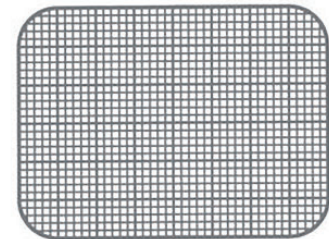

Resin Post Methacrylics

Description

You use the core build upon by the indirect method. It disappears, becomes gas when casting. For the post of the same size and RTP reamer, post selection is easy.

[Specification]

Fine plastic burn - out posts.

Indirect only

Product Code 05000050

Product **Resin Post Methacrylic #R1 (set of 50 pcs)**

Product Code 05000051

Product **Resin Post Methacrylic #R2 (set of 50 pcs)**

Product Code 05000052

Product **Resin Post Methacrylic #R3 (set of 50 pcs)**

#R1(Blue)



#R2(Yellow)



#R3(Red)



Retro Filling Retractor

Description

For reverse root canal treatment, Autoclavable

[Specification]

Material : Stainless steel Measurement length : 176mm

Product Code 30600600

Product **Retro Filling Retractor #1**




Matrix Bands




Matrix Bands

[Specification]
Material : Stainless Steel Thickness : 0.04mm

Product Code	04021101	Product	Matrix Bands #A (set of 100 pcs)
Product Code	04021102	Product	Matrix Bands #B (set of 100 pcs)
Product Code	04021103	Product	Matrix Bands #C (set of 10 pcs)
Product Code	04021104	Product	Matrix Bands #D (set of 10 pcs)
Product Code	04021105	Product	Matrix Bands #E (set of 10 pcs)
Product Code	04021106	Product	Matrix Bands #F (set of 10 pcs)
Product Code	04021107	Product	Matrix Bands #G (set of 10 pcs)
Product Code	04021108	Product	Matrix Bands #H (set of 10 pcs)
Product Code	04021109	Product	Matrix Bands #I (set of 100 pcs)
Product Code	04021110	Product	Matrix Bands #J (set of 100 pcs)

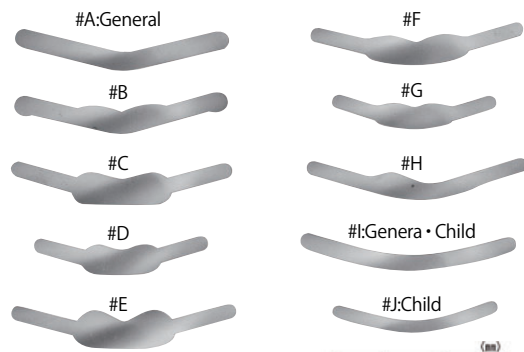

TYPE	Length (mm)	Width (mm)
A	63	6
B	63	7.6
C	65	8
D	51	5
E	66	5
F	64	5
G	49	4.5
H	64	4.5
I	64	5
J	50	4

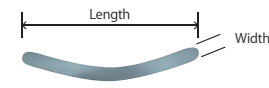


T Bands


Description
Can be used without the Matrix Retainer.

[Specification]
Material : Stainless Steel Thickness : 0.04mm

Product Code	04023111	Product	T Bands, Large (set of 100 pcs)
Product Code	04023112	Product	T Bands, Small (set of 100 pcs)

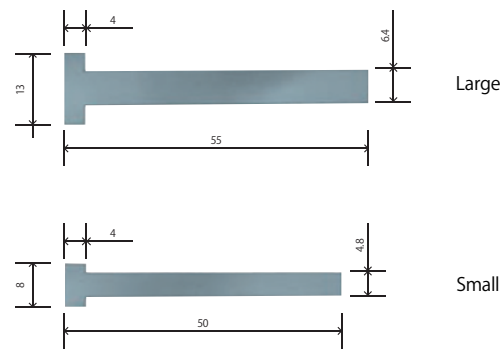

How to use T Bands

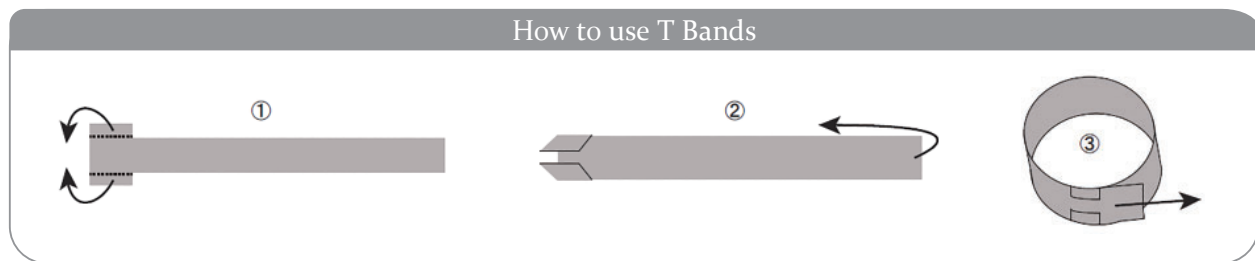

Matrix Retainer #9-T Tofflemire



Description
By a screw that has been improved, it will be a fine movement smooth.

[Specification]
Material : Stainless Steel Size : 64mm

Product Code	04019200	Product	Matrix Retainers Tofflemier Type, Universal Adult, 9-T
--------------	----------	---------	--




Tooth Contact Gauges



Description
For checking interdental contact points.
In three colour coded thicknesses.

[Specification]
Material : Stainless Steel
Holder size:130 x 22 mm
Thickness: Blue 0.05 mm, Yellow 0.11 mm, Red 0.15 mm
Set: Holder x 1pc, Blue Gauges x 3pcs, Yellow Gauge x 1pc, Red Gauge x 1pc

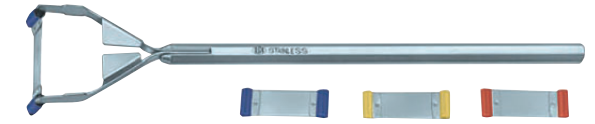

Product Code	50050020	Product	Tooth Contact Gauge Set
Product Code	50050022	Product	Tooth Contact, Refill Gauge Yellow (0.11mm)
Product Code	50050021	Product	Tooth Contact, Refill Gauge Blue (0.05mm)
Product Code	50050023	Product	Tooth Contact, Refill Gauge Red (0.15mm)

New Inlay & Crown Setters



[Specification]
Material : Stainless steel and silicone rubber
Long handle for molars has short pushing pin(SIZE : 79 x 24mm)
Short handle for bicuspids has long pushing pin (SIZE : 69 x 24mm)

Product Code	06092300	Product	New Inlay & Crown Setter Set
Product Code	06092310	Product	New Inlay & Crown Setter I Set
Product Code	06092320	Product	New Inlay & Crown Setter S Set
Product Code	50050035	Product	New Inlay & Crown Setter, Extra Rubber Pads 2 pcs



※Replacement silicone rubber inserts 2pcs.

Tooth Separators

Elliot's Separator



Description
The pushing screw can be attached both right and left side.
Improved jaw design enable to be stable while pushing by the screw.

[Specification]
Material : Stainless Steel
Size : 37 x 67mm

Product Code	01071000	Product	Elliot's Separator
--------------	----------	---------	--------------------



Tooth Separators

Ivory's Separator

Description

With improved curved jaw design, thus separators become stable while pushing by the screw. It is applied in the front area of the upper and lower teeth.

[Specification]

Material : Stainless Steel
Size : 37 x 67mm

Product Code 01072000

Product Ivory's Separator



Articulating Paper Holders

Articulating Paper Holder, M-Type

Description

It is made with milled in order not to move articulating paper and designed to be convenient for the operator.

[Specification]

Material : Stainless Steel
Size : 145mm

Product Code 06121000

Product Articulating Paper Holder, M-Type



Articulating Paper Holder, Y-Type

Description

Sure holding of Articulating papers by pins and holes. Ideal to make for full dentures.

[Specification]


Material : Stainless Steel
Size : Large 74 x 60mm Small 66 x 48mm

Product Code 06122000

Product Articulating Paper Holder, Y-Type, Large

Product Code 06122100

Product Articulating Paper Holder, Y-Type, Small

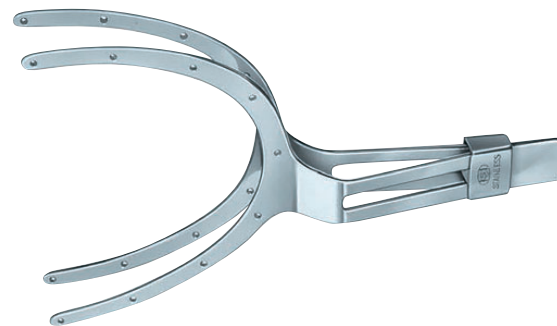


New Fine Lock Trays

Description

"Retention rims" types. Brass, nickel plated. Complete with set of 8 size or individually each size.

Upper					mm
	U11	U12	U13	U14	
Wide	8.5	8.1	7.5	6.8	
Depth	5.9	5.7	5.6	5.4	
Height	1.9	2.0	2.0	2.1	

Lower					mm
	L11	L12	L13	L14	
Wide	8.0	7.6	7.6	7.2	
Depth	6.3	6.5	6.1	6.2	
Height	2.1	2.1	2.2	2.3	

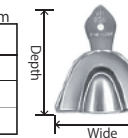

Product Code 06040000

Product New Fine Lock Tray Assort (set of 8 pcs)

Product Code 06041011

Product New Fine Lock Tray #U11

Product Code 06041012

Product New Fine Lock Tray #U12

Product Code 06041013

Product New Fine Lock Tray #U13

Product Code 06041014

Product New Fine Lock Tray #U14

Product Code 06042011

Product New Fine Lock Tray #L11

Product Code 06042012

Product New Fine Lock Tray #L12

Product Code 06042013


Product New Fine Lock Tray #L13

Product Code 06042014

Product New Fine Lock Tray #L14



Dry heat is prohibited

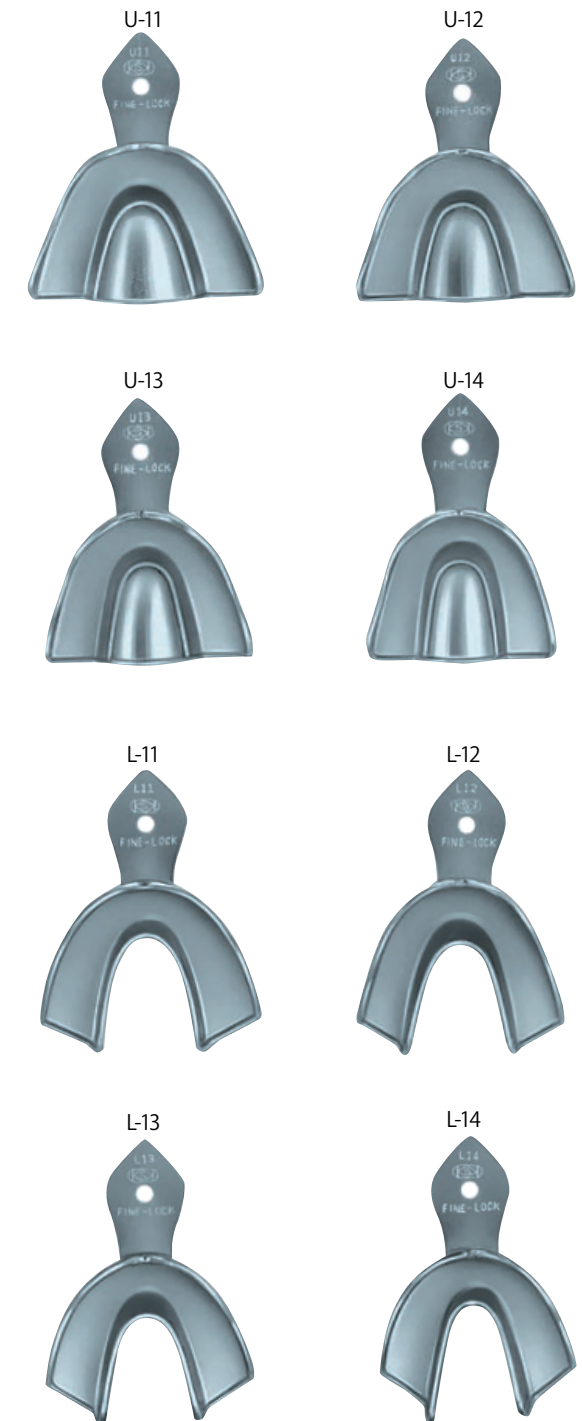



ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

Twist Drills Black

Description

A surgical drill with increased abrasion resistance, corrosion resistance, durability and hardness with DLC coating. It comes with a movable stopper which allows for the length of the drill to be adjusted and a dedicated wrench. The adjustment range for the stopper is between 6 - 11mm for short, and 11 - 16mm for long.

What's DLC?


DLC is short for "Diamond Like Carbon", and it is technology that coats a nano - level hard film with properties that resemble that of diamonds onto metallic surfaces. While having approximately 5 times the hardness compared to existing products, it has shown overwhelming low friction factor and it is recognized as the most technologically advanced among the many surface treatment techniques available.

[Specification]
Material : Stainless steel


Product Code	30073000	Product	Twist Drill "Black" Short 2.0S
Product Code	30073010	Product	Twist Drill "Black" Short 2.6S
Product Code	30073020	Product	Twist Drill "Black" Short 2.8S
Product Code	30073030	Product	Twist Drill "Black" Short 2.9S
Product Code	30073040	Product	Twist Drill "Black" Short 3.0S
Product Code	30073050	Product	Twist Drill "Black" Short 3.1S
Product Code	30073060	Product	Twist Drill "Black" Short 3.2S
Product Code	30073070	Product	Twist Drill "Black" Short 3.3S
Product Code	30073080	Product	Twist Drill "Black" Short 3.4S
Product Code	30073090	Product	Twist Drill "Black" Short 3.5S
Product Code	30073100	Product	Twist Drill "Black" Short 3.6S
Product Code	30073110	Product	Twist Drill "Black" Short 3.7S
Product Code	30073120	Product	Twist Drill "Black" Short 3.8S
Product Code	30073130	Product	Twist Drill "Black" Short 4.0S
Product Code	30073140	Product	Twist Drill "Black" Short 4.2S
Product Code	30073150	Product	Twist Drill "Black" Short 4.5S
Product Code	30073160	Product	Twist Drill "Black" Long 2.0L
Product Code	30073170	Product	Twist Drill "Black" Long 2.8L
Product Code	30073180	Product	Twist Drill "Black" Long 3.1L
Product Code	30073190	Product	Twist Drill "Black" Long 3.2L
Product Code	30073200	Product	Twist Drill "Black" Long 3.3L
Product Code	30073210	Product	Twist Drill "Black" Long 3.5L
Product Code	30073220	Product	Twist Drill "Black" Long 3.8L



Short

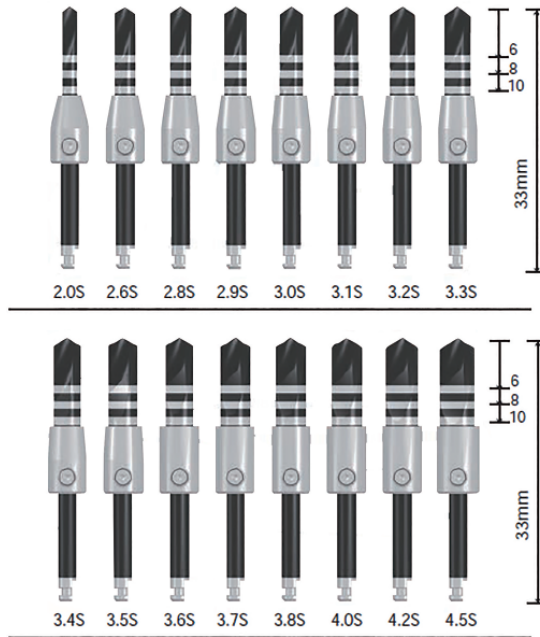

Long

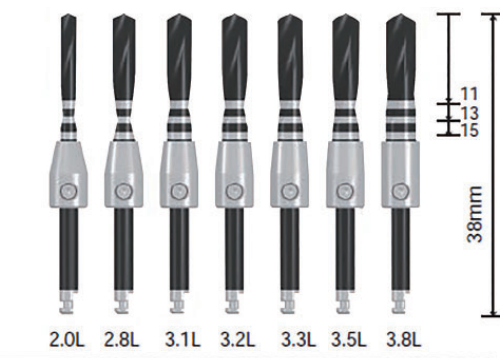


Step Drills For Guide

Description

Simplification of drill step has been made possible. It is ideal for pilot holes for tapered implants.

[Specification]
Material : Stainless steel

Product Code	30075000	Product	Step Drill "Black" for guide ϕ 2.4/2.8mm
Product Code	30075010	Product	Step Drill "Black" for guide ϕ 2.8/3.2mm
Product Code	30075020	Product	Step Drill "Black" for guide ϕ 3.2/3.6mm
Product Code	30075030	Product	Step Drill "Black" for guide ϕ 3.8/4.2mm
Product Code	30075040	Product	Step Drill "Black" for guide ϕ 4.2/4.6mm

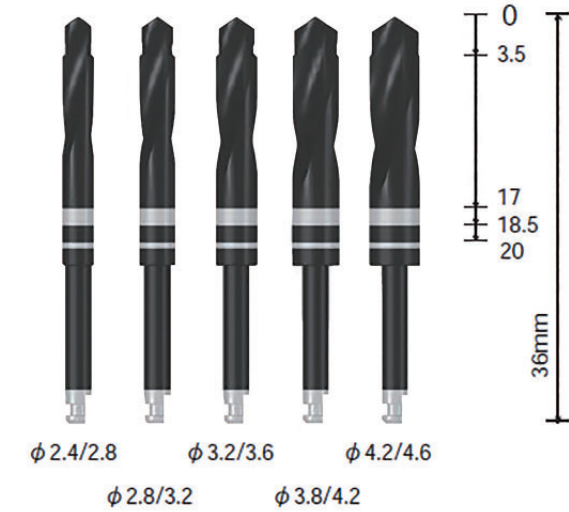


Twist Drills For Guide

Description

When using the 10mm surgical guide, it is marked to drill depths of 7mm, 8.5mm, 10mm.

[Specification]
Material : Stainless steel

Product Code	30074020	Product	Twist Drill "Black" for guide ϕ 2.0mm
Product Code	30074030	Product	Twist Drill "Black" for guide ϕ 3.0mm

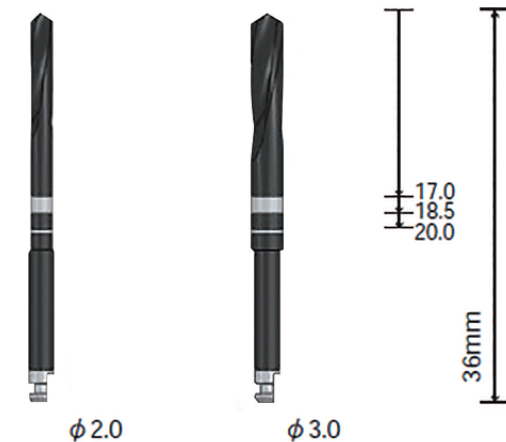

Twist Drills


Twist Drills External Cooling

Description

Our drills are made of stainless steel with high carbon. It assures enough hardness, good cutting efficiency and long life. But rather weak for rust. Careful maintenance will be required to avoid rust and to assure along life.

Good maintenance

- ① Immediately after use, drills should be dipped into anti-rust solution.
- ② Stopper should be removed while cleaning and autoclave.
- ③ Blood and bones should be removed completely after use.
- ④ Use only refined water for autoclave.
- ⑤ Use drill stand while autoclave, do not put into sterilization bag directly.

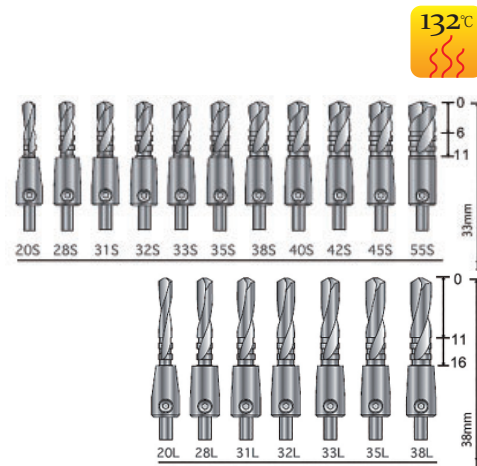

Product Code	30030200	Product	Twist Drill Short 2.0S	Product Code	30030450	Product	Twist Drill Short 4.5S
Product Code	30030280	Product	Twist Drill Short 2.8S	Product Code	30030550	Product	Twist Drill Short 5.5S
Product Code	30030310	Product	Twist Drill Short 3.1S	Product Code	30030201	Product	Twist Drill Long 2.0L
Product Code	30030320	Product	Twist Drill Short 3.2S	Product Code	30030281	Product	Twist Drill Long 2.8L
Product Code	30030330	Product	Twist Drill Short 3.3S	Product Code	30030311	Product	Twist Drill Long 3.1L
Product Code	30030350	Product	Twist Drill Short 3.5S	Product Code	30030321	Product	Twist Drill Long 3.2L
Product Code	30030380	Product	Twist Drill Short 3.8S	Product Code	30030331	Product	Twist Drill Long 3.3L
Product Code	30030400	Product	Twist Drill Short 4.0S	Product Code	30030351	Product	Twist Drill Long 3.5L
Product Code	30030420	Product	Twist Drill Short 4.2S	Product Code	30030381	Product	Twist Drill Long 3.8L


Twist Drills Internal Cooling

Description

The drills with internal cooling system should be used also with external cooling. Drill speed should be controlled from 600rpm(thick drills) up to 1000rpm(for fine drills).

Cutting efficiency is coming bad by each use and bad drill will cause heat damages against bone. Only good drills should be used for operation. For very hard bones, it is recommended to use surgical burs prior to implant drills. Surgical burs will help easy drilling and long life. The fix screw of drill stopper is so small and it can not be bear a strong force. Drills can be used also without stopper.

[Specification]
Material : Stainless steel

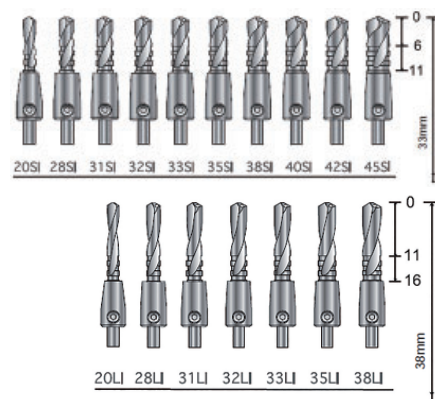


Product Code	30031200	Product	Twist Drill Short 2.0 SI	Product Code	30031201	Product	Twist Drill Long 2.0 LI
Product Code	30031280	Product	Twist Drill Short 2.8 SI	Product Code	30031281	Product	Twist Drill Long 2.8 LI
Product Code	30031310	Product	Twist Drill Short 3.1 SI	Product Code	30031311	Product	Twist Drill Long 3.1 LI
Product Code	30031320	Product	Twist Drill Short 3.2 SI	Product Code	30031321	Product	Twist Drill Long 3.2 LI
Product Code	30031330	Product	Twist Drill Short 3.3 SI	Product Code	30031331	Product	Twist Drill Long 3.3 LI
Product Code	30031350	Product	Twist Drill Short 3.5 SI	Product Code	30031351	Product	Twist Drill Long 3.5 LI
Product Code	30031380	Product	Twist Drill Short 3.8 SI	Product Code	30031381	Product	Twist Drill Long 3.8 LI
Product Code	30031400	Product	Twist Drill Short 4.0 SI	Product Code	30032000	Product	Screw for Implant Drill Stopper
Product Code	30031420	Product	Twist Drill Short 4.2 SI	Product Code	30032001	Product	Hexagonal Wrench for Implant Drill Stopper
Product Code	30031450	Product	Twist Drill Short 4.5 SI				

Trephine Burs

Description

Trephine Burs do not have blade at center, thus it will be rather difficult to get smooth cutting for a start. Enough exercise is recommended before use. After Trephine Burs penetrate into bone, then smooth cutting is available. To avoid choking on bone inside Trephine Bur, pull of Trephine Burs up and down, so that bone will always remain at patient. Remained bone can be collected by surgical curettes. Enough cooling and possibly low speed is recommended to avoid heat damage against bone.

[Specification]
Material : Stainless steel

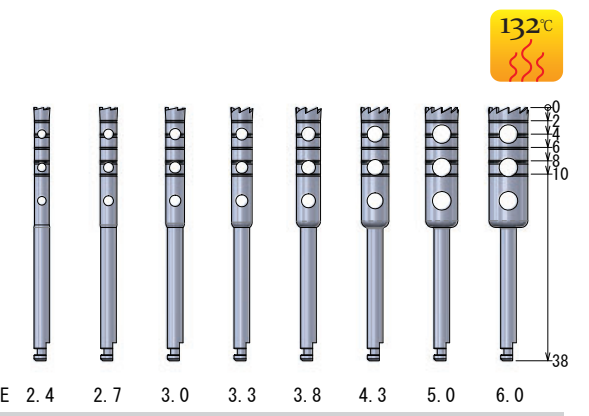


Product Code	30060240	Product	Trephine Bur 2.4	Product Code	30060380	Product	Trephine Bur 3.8
Product Code	30060270	Product	Trephine Bur 2.7	Product Code	30060430	Product	Trephine Bur 4.3
Product Code	30060300	Product	Trephine Bur 3.0	Product Code	30060500	Product	Trephine Bur 5.0
Product Code	30060330	Product	Trephine Bur 3.3	Product Code	30060600	Product	Trephine Bur 6.0

Bone Trephines

Description

Bone Trephine is used for collection of big volume of bone. Enough exercise is recommended before use.

[Specification]
Material : Stainless steel

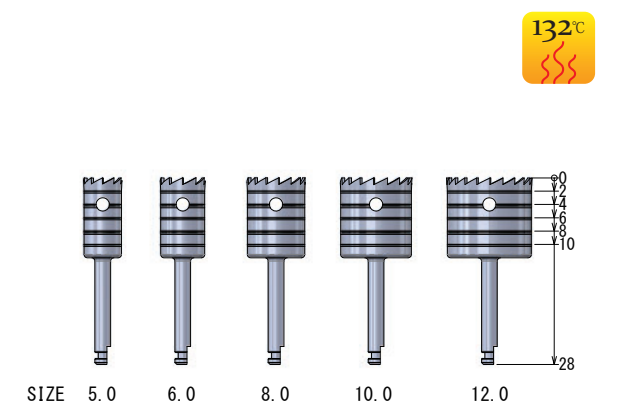


Product Code	30070500	Product	Bone Trephine 5.0
Product Code	30070600	Product	Bone Trephine 6.0
Product Code	30070800	Product	Bone Trephine 8.0
Product Code	30071000	Product	Bone Trephine 10.0
Product Code	30071200	Product	Bone Trephine 12.0

Circular Knives

Description

For cutting soft tissue. Use with very low speed or by finger. Complete with finger grip for each piece.

[Specification]
Material : Stainless steel

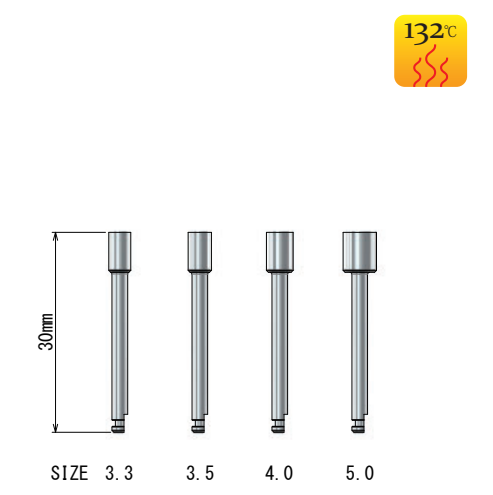

Product Code	50010000	Product	Circular Knife 3.3 without finger driver
Product Code	50010010	Product	Circular Knife 3.5 without finger driver
Product Code	50010020	Product	Circular Knife 4.0 without finger driver
Product Code	50010030	Product	Circular Knife 5.0 without finger driver
Product Code	30051001	Product	Finger Driver for Circular Knife

ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS





Surgical Burs

Surgical Bur External Cooling System

Description

- Round Burs
Round burs for bone cutting, ruff cutting edges to prevent clogging. Full group blades ensures excellent cutting efficiency.
- Dr. Lindemann Bur
Dr. Lindemann Bur has fully around blades.

Product Code	30011400	Product	Surgical Bur Round 141RF/010
Product Code	30011401	Product	Surgical Bur Round 141RF/014
Product Code	30011402	Product	Surgical Bur Round 141RF/018
Product Code	30011411	Product	Surgical Bur Round 141RF/025
Product Code	30011412	Product	Surgical Bur Round 141RF/031
Product Code	30011414	Product	Surgical Bur Round 141RF/040
Product Code	30011415	Product	Surgical Bur Round 141RF/050
Product Code	30011631	Product	Dr. Lindemann Bur 163RF/014
Product Code	30011641	Product	Dr. Lindemann Bur 164RF/018
Product Code	30011661	Product	Dr. Lindemann Bur 166RF/021

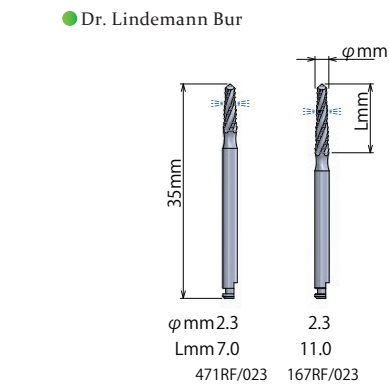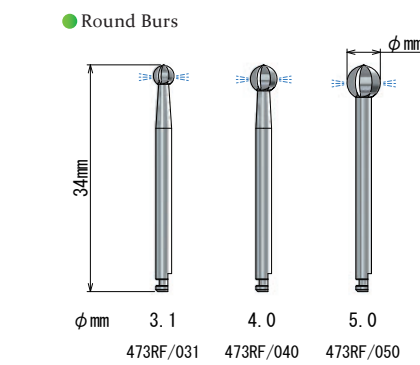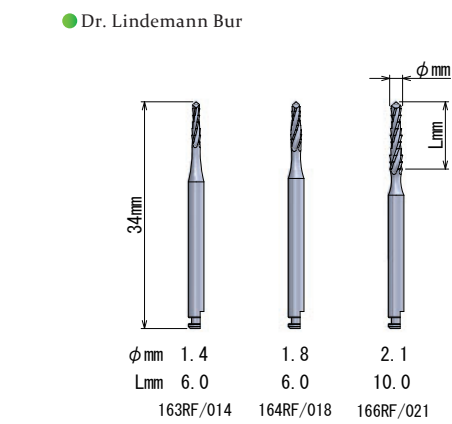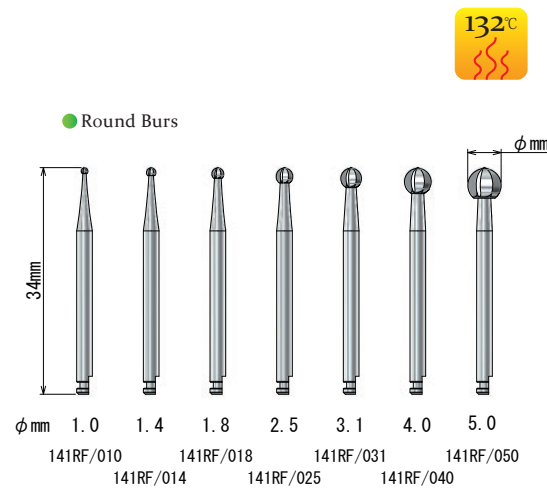

Surgical Bur Internal Cooling System

Description

These hully group surgical burs with internal cooling system enable very smooth cut and prevent heat damages for bone. Cooling fluid flow out blood thus operator can get clear view of operating spot. Prepare #40 K Reamers to clean internal cooling hold.

Product Code	30014711	Product	Surgical Bur Internal Cooling 471RF/023
Product Code	30011671	Product	Surgical Bur Internal Cooling 167RF/023
Product Code	30014731	Product	Surgical Bur Internal Cooling 473RF/031
Product Code	30014732	Product	Surgical Bur Internal Cooling 473RF/040
Product Code	30014733	Product	Surgical Bur Internal Cooling 473RF/050

Technika TK Drill System

Description

Innovative dental drill for implant treatment.
Safe and Secure drill systems
Completed with a minimum 4drills, easy steps
Ultra-low speed rotation, no water injection
Autologous bone guide / Autologous bone collection

[Specification]

- Contents : ① Surgical Bur Round 141RF/018
② Twist Drill Short 2.0S (External Cooling)
③ Technika TK Countersink (3.0, 3.5, 4.0, 4.5, 5.0)
④ Technika TK Drill (φ3.0, φ3.5, φ4.0, φ4.5, φ5.0)
⑤ Technika Surgical Cassette *final drill is not included

Product Code	30120000	Product	Technika TK Drill System
--------------	----------	---------	--------------------------

- ① Surgical Bur Round 141RF/018
Material : Stainless Steel
Length : 34 mm,
Size : φ1.8mm
Quantity : 1pc


- ② Twist Drill Short 2.0S (External Cooling)
Material : Stainless Steel
Length : 33 mm,
Size : φ2.0
Quantity : 1pc

- ⑤ Technika Surgical Cassette
Material : Resin and Silicone
Size : 99.5 mm x 143 mm x 61 mm
Quantity : 1pc

- ③ Technika TK Countersink
Material : Stainless Steel
Length : 24 mm,
Size : 3.0, 3.5, 4.0, 4.5, 5.0
Quantity : 1pc each

- ④ Technika TK Drill
Material : Stainless Steel
Length : 36 mm,
Size : φ3.0, φ3.5, φ4.0, φ4.5, φ5.0
Quantity : 1pc each

● included drills

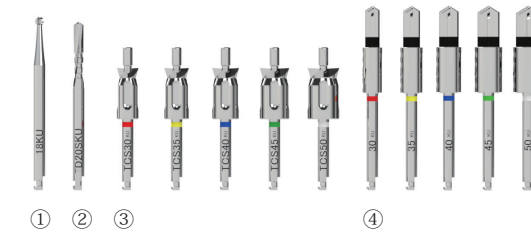


Implant Drill Checker

Description

Can be measured diameter of 2.0 - 5.9mm.
Can be measured length of 6.0 - 18.0mm.
With length scale of 5 - 20mm. Can be used for drill stand.

[Specification]
Material : Stainless steel

Product Code 30080012
Product Implant Drill Checker


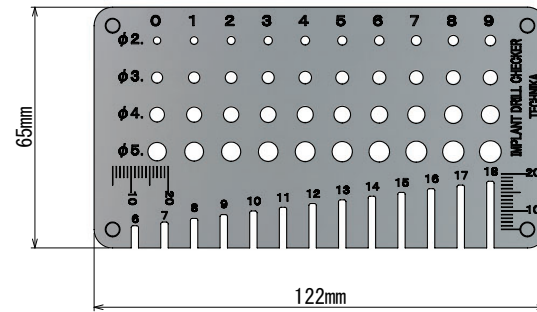


Drill Extenders

Description

Two types : External cooling/Internal cooling
Extension length : External cooling is 16.5mm,
Internal cooling is 18 mm.

[Specification]
Material : Stainless steel

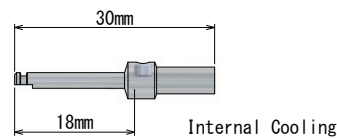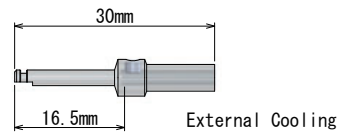

Product Code 30080001
Product Drill Extension External Cooling

Product Code 30080002
Product Drill Extension Internal Cooling

Drill Extension SHORT External Cooling

Description

Can extend the drill length slightly 5mm.
Easy attaching without tools.

[Specification]
Material : Stainless steel Diameter : 7.7 mm

Product Code 30079991
Product Drill Extension SHORT External Cooling




Para Drill Aid II

Description

- Attachment drill aid for parallel implantation.
- Four guid - pins and three drill - sleeves make surgery better and shorter.
- You can easily make a hole for parallel implantation.
- Pilot drill (dia 1.8 to 2.0mm) is available.
- All parts are autoclavable.

[Specification]
Material : Stainless steel

Product Code 30080200
Product Para Drill Aid II

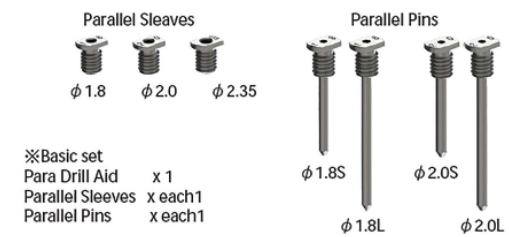

Technika Bone Mill Set

Description

- Simple and compact
- Easy operating
The angle mount enables easy and smooth operating.
- Easy milling
The unique shaped cutter with DLC coating enable easy and fine milling.
- Easy assembling
The simple structure saves time for operating and cleaning With sterilization case and cleaning brush Cutter for substitute. ※sell separately

Product Code 30080015
Product Technika Bone Mill

Product Code 30080016
Product Replacement Cutter

Product Code 30080117 <STOCK ONLY>
Product Technika Bone Mill with sterilization case





Technika Repair Tap

Description

- For internal threads repair.
- Made of high corrosion resistant stainless steel compatible with autoclave.
- Long lasting DLC coating.
- Safe Handle

[Specification]

Material : Stainless steel (Tap and Handle)
 Surface Treatment : DLC coating (Tap)
 Thread Diameter x Pitch : M1.6 x 0.35 / M2.0 x 0.4
 Made in Japan

Product Code	30011110	Product	REPAIR TAP M1.6 with Handle
Product Code	30011130	Product	REPAIR TAP M2.0 with Handle
Product Code	30011210	Product	M1.6 Tap only
Product Code	30011230	Product	M2.0 Tap only
Product Code	30011300	Product	Handle for Repair Tap

● Repair Tap with Handle(Basic Set)



● Repair Tap



● Handle for Repair Tap



Compatible to both Tap


Depth Gauges

Description

Graduations on 8, 10, 13, 15, 18, 20.
 Graduations can be exposed on X-ray film.
 Diameter : 1.3, 1.5, 1.9, 2.7, 3.0, 3.2, 3.4, 3.9mm.

[Specification]

Material : Stainless steel

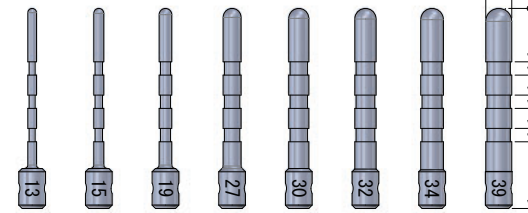

Product Code	30072130	Product	Depth Gauge 1.3
Product Code	30072150	Product	Depth Gauge 1.5
Product Code	30072190	Product	Depth Gauge 1.9
Product Code	30072270	Product	Depth Gauge 2.7

Product Code	30072300	Product	Depth Gauge 3.0
Product Code	30072320	Product	Depth Gauge 3.2
Product Code	30072340	Product	Depth Gauge 3.4
Product Code	30072390	Product	Depth Gauge 3.9

3.75 Special Drills

Description

Diameter is 3.75mm.
 Twist drill with safety stopper.

[Specification]

Material : Stainless steel



Product Code	30043751	Product	φ3.75 Special Drill φ2.0 x 10
Product Code	30043752	Product	φ3.75 Special Drill φ2.0 x 13
Product Code	30043753	Product	φ3.75 Special Drill φ2.0 x 15


Product Code	30043754	Product	φ3.75 Special Drill φ3.1 x 10
Product Code	30043755	Product	φ3.75 Special Drill φ3.1 x 13
Product Code	30043756	Product	φ3.75 Special Drill φ3.1 x 15

Gingiva Gauge

Description

The first division shows 1mm when top of measuring block and needle are coincide. Wide of graduation is 1mm and pitch is 2mm, thus each graduation shows the gingival thickness of 2, 3, 4, 5, 6, 7mm. Put the measuring block in right angle to gingiva, then prick gingival by light force until needle reaches to bone. Pull out carefully so that measuring block would not moves. Read the thickness at the back side of measuring block.

Product Code 30080013 Product Gingiva Gauge

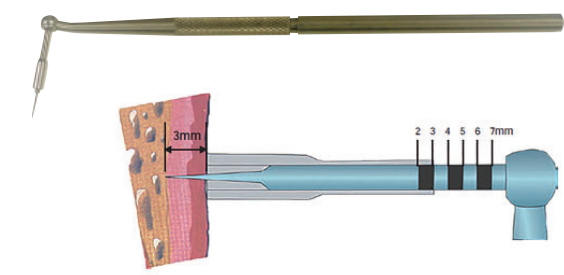

- Can be measured the thickness of gingiva from 2mm to 7mm in increments of 1mm during implant operation.
- For a more exact and detailed measure. It is better to use micrometer caliper separately.

Sterilization Box

<STOCK ONLY>

Description

- Sterilization Box
Can be housed three bur stands.

[Specification]
 Material : Aluminum
 Size : 170 x 70 x 30mm

● Sterilization Box




Bur Stands

Description

You can stand 6 burs.

[Specification]
 Material : Stainless Steel
 Bur Stand 40(w70 x D40 x H15mm) Compliant bur length : Max 35mm
 Bur Stand 45(w70 x D45 x H15mm) Compliant bur length : Max 40mm

Product Code 30080010 Product Bur Stand 40
 Product Code 30080011 Product Bur Stand 45

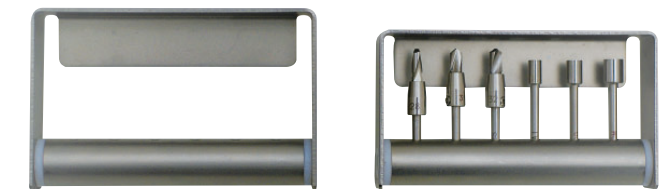


Lock Stand

Description

No falling off, burs keep stable.

[Specification]
 Material : Stainless Steel , Autoclavable
 Lock Stand : w70 x D53 x H15mm, Bur/drill size : up to 45mm, φ 2.35mm

Product Code 30600500 Product Lock Stand

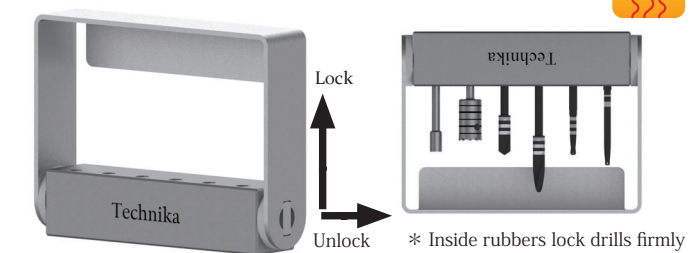

* Inside rubbers lock drills firmly

ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

Vulcanite Burs


[Specification]

Material : Stainless Steel

Large and small types are available.

Large : Maximum diameter is 6 mm.

Small : Maximum diameter is 4.8 mm.

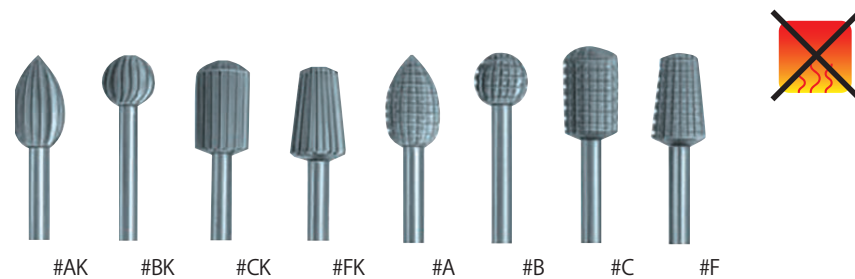

※ Straight hand piece type only.

Product Code	02101011	Product	Vulcanite Burs #A, LARGE	Product Code	02111011	Product	Vulcanite Burs #AK, LARGE
Product Code	02101012	Product	Vulcanite Burs #A, SMALL	Product Code	02111012	Product	Vulcanite Burs #AK, SMALL
Product Code	02101021	Product	Vulcanite Burs #B, LARGE	Product Code	02111021	Product	Vulcanite Burs #BK, LARGE
Product Code	02101022	Product	Vulcanite Burs #B, SMALL	Product Code	02111022	Product	Vulcanite Burs #BK, SMALL
Product Code	02101031	Product	Vulcanite Burs #C, LARGE	Product Code	02111031	Product	Vulcanite Burs #CK, LARGE
Product Code	02101032	Product	Vulcanite Burs #C, SMALL	Product Code	02111032	Product	Vulcanite Burs #CK, SMALL
Product Code	02101061	Product	Vulcanite Burs #F, LARGE	Product Code	02111061	Product	Vulcanite Burs #FK, LARGE
Product Code	02101062	Product	Vulcanite Burs #F, SMALL	Product Code	02111062	Product	Vulcanite Burs #FK, SMALL


Air Jet Diamonds

Features

Air Jet Diamond have air slits at cutting head, the slits supply strong air while cutting.

The strong air flow prevent the clogging of cutting materials. Thus, you can very smoothly and effective cutting.

Product Code	02091001	Product	Air Jet Diamonds #1
Product Code	02091002	Product	Air Jet Diamonds #2
Product Code	02091003	Product	Air Jet Diamonds #3
Product Code	02091010	Product	Air Jet Diamonds #1A
Product Code	02091011	Product	Air Jet Diamonds #2A

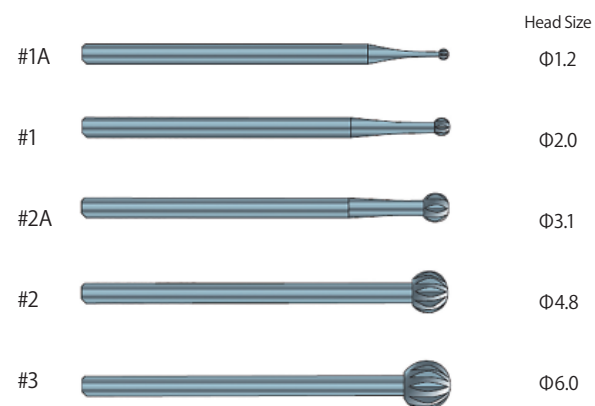

Lip & Cheek Retractor

Features

To retractor cheeks and to have a better view.

[Specification]

Material : Polycarbonate Set of 2pcs

Product Code 50570131 Product Lip & Cheek Retractor Set



121°C
Dry heat is prohibited

Dry Field Mouth Props <STOCK ONLY>

Features


Great for air - abrasion prep and sealants .

- Comfortably holds the jaw open.
- Retracts and protects the patients cheek and tongue.
- Create a moisture - free environment.

[Specification]

Material : Polycarbonate

Product Code	50570000	Product	Dryfield Mouth Props #1 labial (Set of 3 pcs)
Product Code	50570002	Product	Dryfield Mouth Props #2 large (Set of 3 pcs)
Product Code	50570001	Product	Dryfield Mouth Props #3 small (Set of 3 pcs)
Product Code	50570005	Product	Dryfield Mouth Props Assort (#1,2,3)

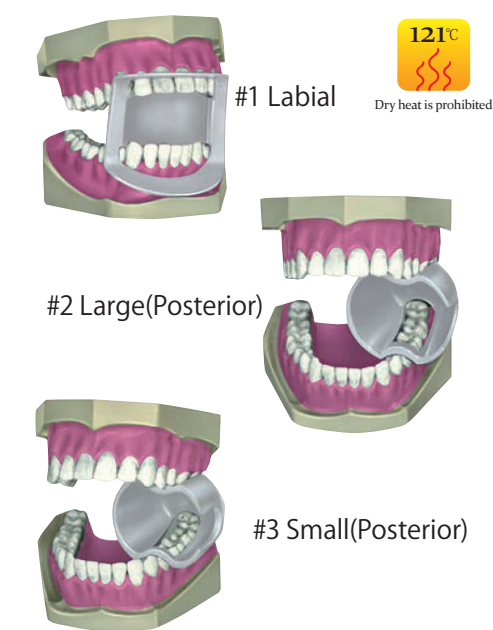

121°C
Dry heat is prohibited

Lip Expanders

Features

Superb anterior access. Available in an adult, child or assorted(adult / child)set.

- Soft, plastic frame.
- Comfortable for the patient.
- Provides excellent anterior access.

[Specification]

Material : Polypropylene

Product Code	50570010	Product	Lip Expander Adult Size (set of 2 pcs)
Product Code	50570011	Product	Lip Expander Child Size (set of 2 pcs)
Product Code	50570020	Product	Lip Expander Adult & Child, (set of 2 pcs)



121°C
Dry heat is prohibited



Bite Expander <STOCK ONLY>

Description

"Bite Expander" has function of Retractor and Bite block. You can easily change 4 patterns for every treated area.

- This device serves the purpose of both a cheek retractor and bite block.
- Can be used on every part of the mouth.
- Made from silicone and polypropylene.

[Specification]


Material : Silicon rubber and Polypropylene

Product Code 07000150

Product **Bite Expander**

Product Code 07000155

Product **Extra Rubber Pads for Bite Expander (2pcs)**

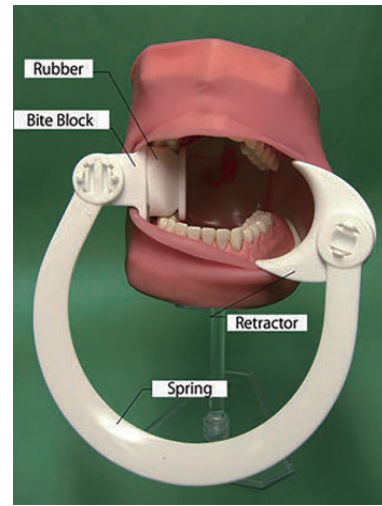

Dry heat is prohibited

Big Oral Mirrors Real-Fine

Features

- Ideal for distortion - free intra - oral photos.
- Extensive rhodium coating on one side.
- Sturdy, durable.
- With detachable silicone handle.
- Autoclavable in the provided fabric bag.
- Reflection rate : 73%

#1 : Side views. #2 : Occulusal views (medium). #3 : Occulusal views (large).

#4 : For lingual side views of posteriors.

#5 : For lingual side views of anteriors. Handle for #4 , #5 : aluminum.

Big Oral Mirror Set : Assort of #1,#2 and #3

Product Code 07000100

Product **Big Oral Mirror Set (#1~#3)**

Product Code 07000101

Product **Big Oral Mirror #1**

Product Code 07000102

Product **Big Oral Mirror #2**

Product Code 07000103

Product **Big Oral Mirror #3**

Product Code 07000104

Product **Silicone Handle for Big Oral Mirror**

Product Code 07000105


Product **Big Oral Mirror #4**

Product Code 07000106

Product **Big Oral Mirror #5**

Product Code 07000109

Product **Aluminium Handle for Big Oral Mirror**

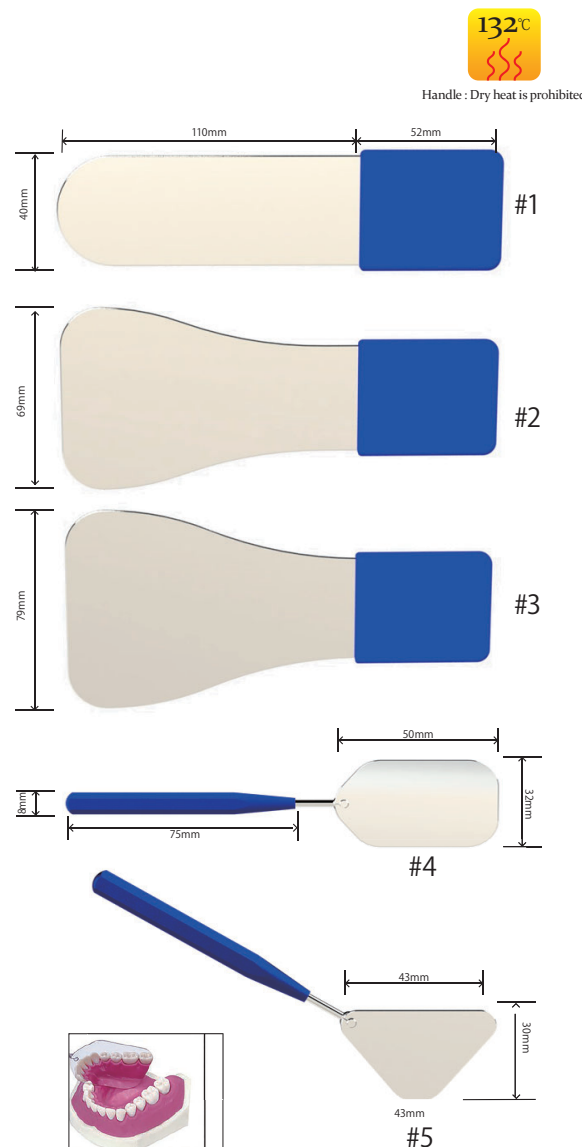

Cotton Pliers

Description


Available for picking up 0.01g.

[Specification]

Coak type Material : Stainless steel Size : 162mm

Product Code 06301010

Product **Cotton Plier**

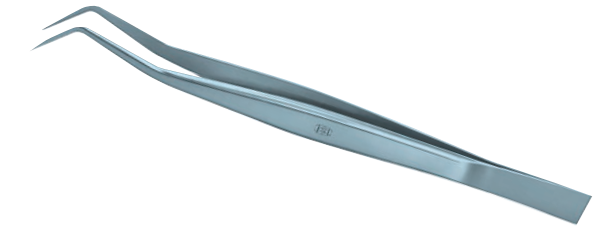

Tongue Guards & Saliva Ejector

Description

Three different sizes metal reflectors are changeable and they protect tongue from turbine.

[Specification]

Pipe : brass, chrome plated Others : Stainless steel Size : 138 x 80mm

Product Code 50050010

Product **Tongue Guards & Saliva Ejector**



MOQ : 100pcs. Please contact us about the price every order.

Parallel Reflector

Features


Mirror with parallel lines. Autoclavable.

[Specification]

Coak type Material : Stainless steel Size : H 18 x W 48 mm, Thread Part M2.5 (P=0.45 L-5)

Product Code 50050100

Product **Parallel Reflector**

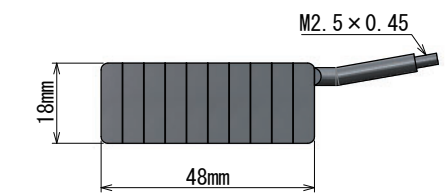

MOQ : 100 pcs



ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

ENDODONTICS

PROSTHETICS

IMPLANTOLOGY

LABORATORY

MISCELLANEOUS

pinlight2


Features

- The mirror and the light guide are detachable and autoclavable.
- The handle is IP X7 waterproof and capable of alcohol disinfection.
- Much brighter 90lm White LED Lighting used for detecting caries.
- Roder's HR FRONT mirror comes with size 4.

[Specification]

Light : 90LM, Cool white LED
 Mirror : HR FRONT coating, size 4
 Battery : Two AAA-size
 Waterproof Performance IPX7 JIS C 0920 (IEC 60529)
 Continuous lighting time : 90 min

Product Code	30080350
Product	pinlight2
Product Code	30080420
Product	Mirror holder, set of 4pcs.
Product Code	30080360
Product	Light guide
Product Code	30080370
Product	Replacement O ring, set of 3pcs.

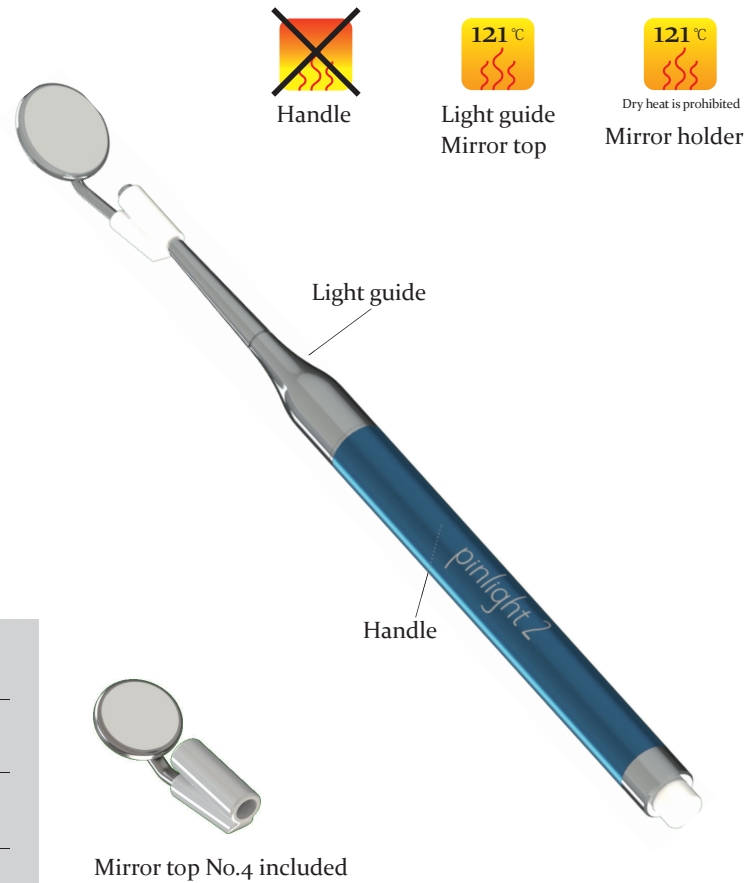

Handle



Light guide
Mirror top



Mirror holder

Excavators

Features

special hollow octagon handle which enables very light weight as well as ideal balance.
 high cutting performance and long life

[Specification]

Material : Stainless Steel Size : 165 mm x Handle : 5.8 mm

Product Code 50581100 Product Excavators #19/20

Product Code 50581150 Product Excavators #21/22

#19/20



#21/22



MOQ : 100 pcs

Wax Spatulas #7A

[Specification]

Material : Stainless Steel Size : 150 mm x Handle : 4.5 mm

Product Code 50581020 Product Wax Spatulas #7A



MOQ : 100 pcs





ENDODONTICS

Evans

[Specification]
Material : Spoon Stainless Steel, Handle Aluminum Size : 171 mm x Handle : 8 mm

Product Code 50581000 Product Evans



MOQ : 100 pcs

PROSTHETICS

Hollenback #3

[Specification]
Material : Stainless Steel Size : 166 mm x Handle : 5.8 mm

Product Code 50581040 Product Hollenback #3



MOQ : 100 pcs

IMPLANTOLOGY

Plastic Filling Instruments, Woodson #1

[Specification]
Material : Stainless Steel Size : 174 mm x Handle : 5.8 mm

Product Code 50581060 Product Plastic Filling Instruments, Woodson #1



MOQ : 100 pcs

LABORATORY

MISCELLANEOUS



TEL:+81-3-3964-2011

FAX:+81-3-3962-5624

53-5, Shimizu-cho, Itabashi-ku,
Tokyo, 174-0053, JAPAN

URL www.dentech.co.jp

MAIL info@dentech.co.jp

